


## THE STOPPER

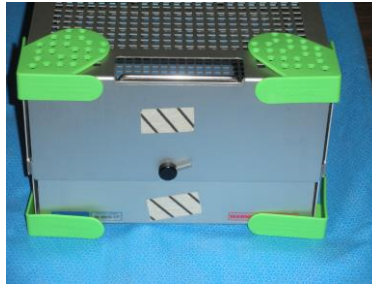

### Instructions for Use

Place surgical tray on top of desired sterile wrap

1. Lift each bottom corner of surgical tray and slide the STOPPER under the foot or corner of tray

2. Make sure the edge guards fit up against the surgical tray

The edge guards will not fit flush against the tray because of the dimples/studs inside of the edge guards

These dimples/studs allow for complete circulation of the sterilizing agent into the corners of the surgical tray

3. Repeat 2 above only invert the STOPPER to the top corners of the tray

After the STOPPER has been applied to all corners, continue to wrap the surgical tray as per hospital /Clinic/Surgery Centre protocol

### Instructions for Cleaning

Point of use:

Remove soil with disposable cloth or paper wipe

### Cleaning

Remove all visible soil and contaminates using soft bristle brush

The entire STOPPER should be immersed while cleaning to aid the in removal of contaminates and to reduce splashing of detergents on personnel

Rinse thoroughly with clean water to remove all detergent.

Automated washer disinfecter process

After completion of automatic washer cycle, visually inspect the STOPPER for any remaining soil

All visible soil must be removed by hand cleaning, brushing, and additional automatic cycle prior to being used.

 DISTRIBUTED BY KEY SURGICAL  
5101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA  
(US) PH: 800.541.7995 KEYSURGICAL.COM  
(UK) TEL: +44 (0) 1628 810626 KEYSURGICAL.CO.UK  
(EU) TEL: +49 4565 905900 KEYSURGICAL.DE