

Name: Choice Together Sterilisation Wrap

Product Code: CT43LGT-xxx, CT50MED-xxx, CT60HVY-xxx

Name of Manufacturer:



Intended Use

The intended use of Choice Together Sterilisation Wrap is as a microbial barrier sterilisation wrap for use in sterile barrier packaging systems which are designed for the sterilisation of surgical instruments by steam, ethylene oxide or gamma radiation methods.

Choice Together Sterilisation Wrap is intended to be used by professional healthcare personnel working in sterile services (or similar) departments.

Single Use

Choice Together Sterilisation Wrap is single use only.

Sterilisation Compatibility

Choice Together Sterilisation Wrap is compatible to steam, Ethylene Oxide, Formaldehyde and Hydrogen Peroxide sterilisation techniques.

Storage Conditions

It is recommended that Choice Together Sterilisation Wrap is stored under the conditions stated on the label of the box. If this is not possible, it should be prevented from getting wet or from being exposed to high temperatures (>40degC).

Precautions


Choice Together Sterilisation Wrap should be stored away from any sources of ignition.

Wrapping with Choice Together Sterilisation Wrap


- 1) Choice Together Sterilisation Wrap can be used in the same way as any other standard Tray Wrap.
- 2) Any of the tray wrapping techniques can be used in either a sequential or simultaneous wrapping procedure.

Below are examples of 2 well-known wrapping techniques - Parcel Fold and Envelope fold.

Parcel Fold

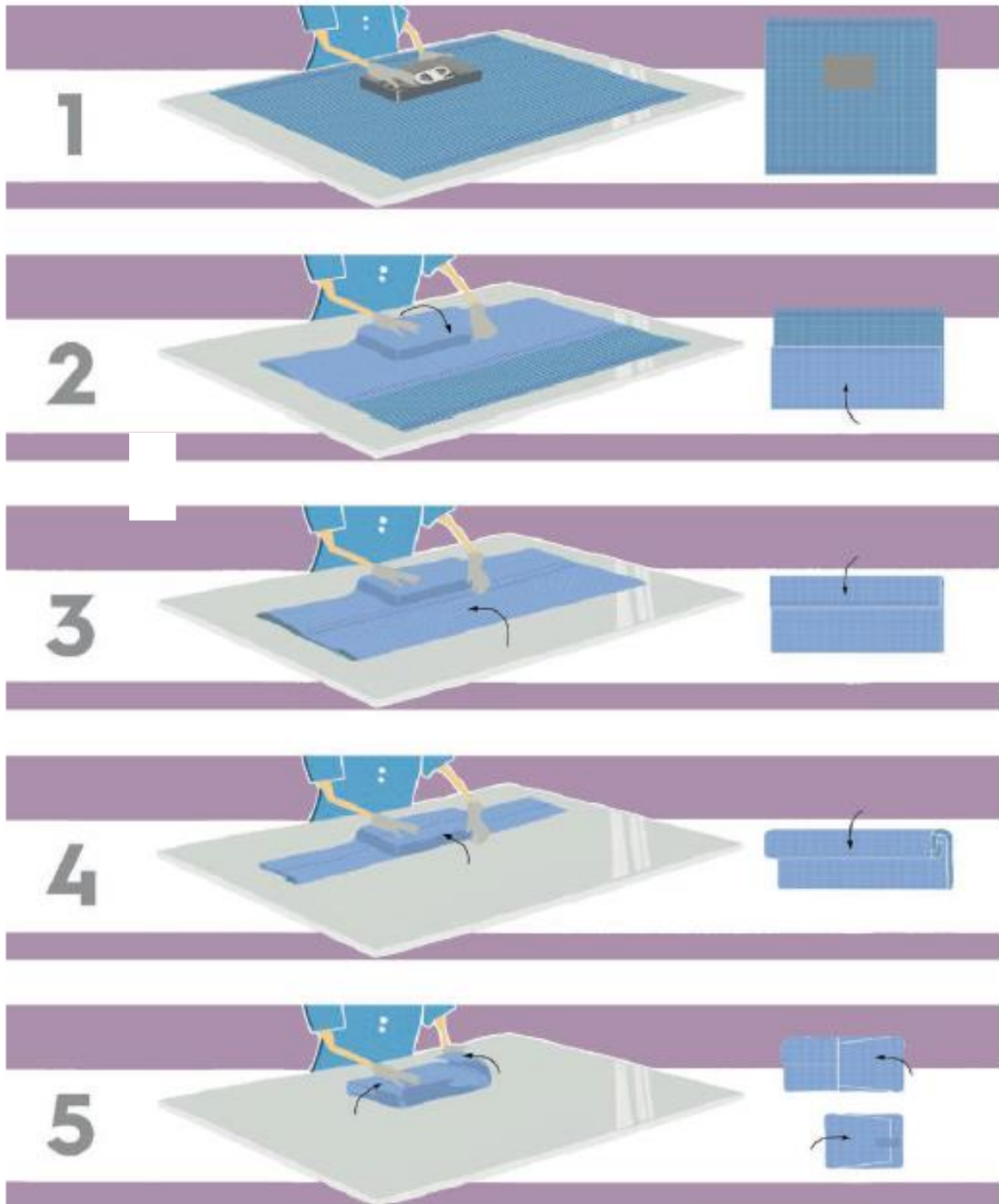

Envelope Fold

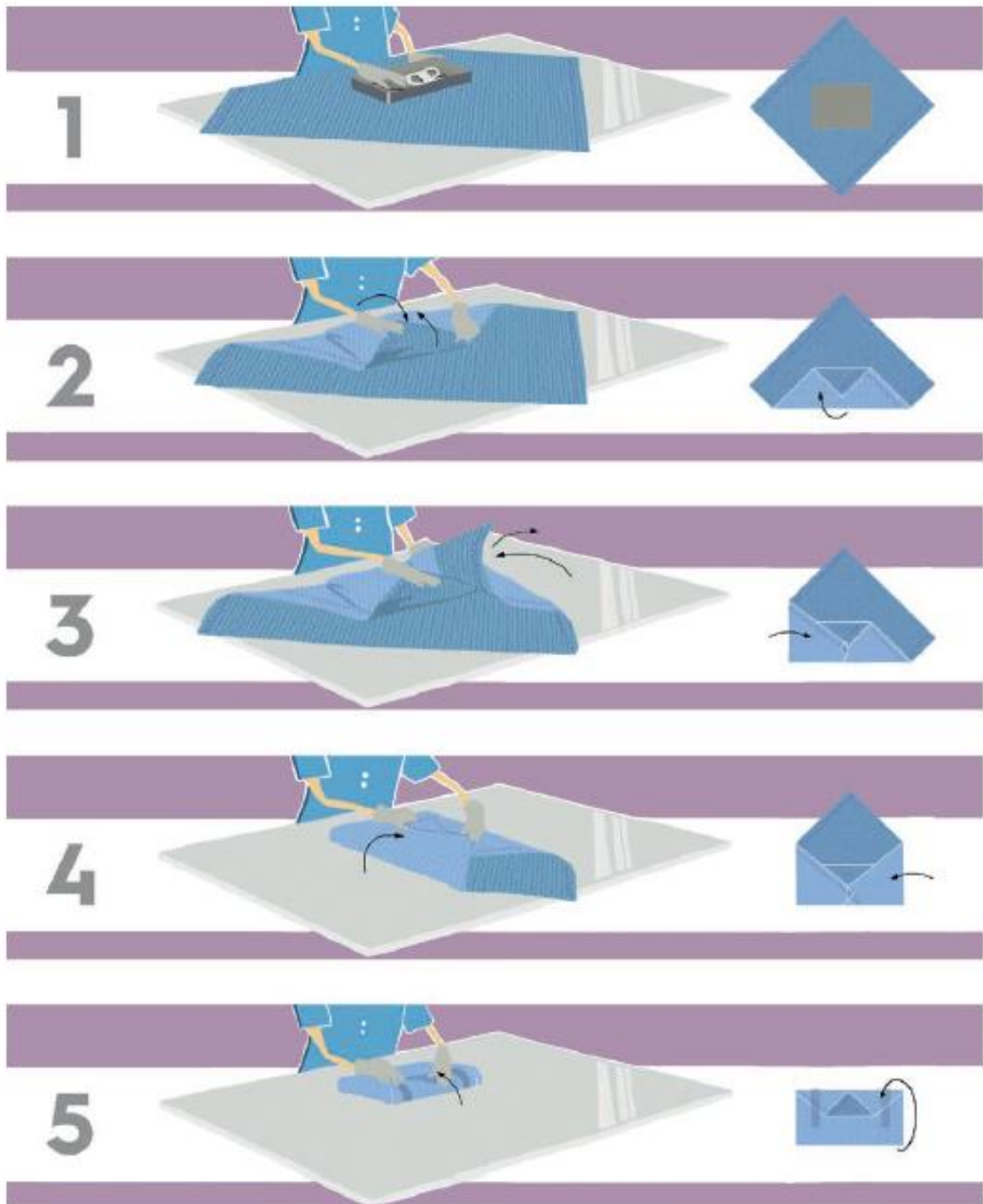

Securing the Wrapped Package

Secure with an appropriate closure (tape or alternate closure suitable for the sterilization method to be applied) and label. Closure must:

- Allow the sterilant to penetrate the wrapped package
- Avoid constriction of the package
- Maintain package integrity

Disposal

Unused Choice Together Sterilisation Wrap may be disposed of in the same way that the user normally disposes of wrap. It is suitable for recycling in the waste wrap stream. Used Choice Together Sterilisation Wrap should be disposed of by considering possible microbial contamination. The users should reference their own procedures in this matter.