

Name: Millennium Blue Tray Liners & Blue Heavy Tray Liners

Product Codes: MILBxxx, MILBxxxXxxx, LIBxxxXxxx

Name of Manufacturer:



### Intended Uses

Millennium Blue Tray Liners are bonded viscose sheets which are intended to be used as tray liners to allow condensate to be quickly dispersed during steam sterilisation of medical instruments, thus reducing the frequency and “wet packs”. They are supplied in a range of sizes designed to fit the sterilisation trays in common use.

Blue Heavy Tray Liners are manufactured from a heavy non-woven fabric and can be used as a shelf liner for added protection from rough handling and wet packs. The nonwoven fabric maintains strength and uniformity, even when wet. Its abrasion resistance combined with a high level of absorbency makes the fabric the best compromise for tray protection. Blue heavy liners can be used under trays to not only protect inner wraps from the sharp corners or feet on trays and baskets, but to absorb condensate during sterilisation. The sheets can be used inside a tray or more commonly used under a tray to protect sterilisation wraps from internal damage and subsequent handling.

Blue Tray Liners are intended to be used by professional healthcare personnel working in sterile services (or similar) departments and are single-use only.

### Sterilisation Compatibility

Millennium Blue Tray Liners are suitable for use in steam (up to 137°C) and ethylene oxide sterilisation processes. They are not suitable for use in plasma sterilisation processes.

Blue Heavy Liners are compatible with steam sterilisation processes up to 137°C for 3.5 minutes and Ethylene Oxide procedures up to a temperature of 65°C. They are not compatible with plasma sterilisation procedures.

### Single use

Blue Tray Liners are single use only.

### Storage Conditions

It is recommended that Blue Tray Liners are stored under the conditions stated on the label of the box. If this is not possible, they should be prevented from getting wet or from being exposed to high temperatures (>40degC).

## Precautions

Blue Tray Liners should be stored away from any sources of ignition.

## Disposal


Unused Blue Tray Liners are only recyclable through specialised outlets. If these are not available, they should be disposed of to Landfill.

Used Blue Tray Liners should be disposed of by considering possible microbial contamination. Users should reference their own procedures in this matter.

## Instructions for use

### Millennium Tray Liner Sheets

- 1) Choose the most appropriate size for the basket to be used.
- 2) Place the Millennium Blue Tray Liner in the bottom of the basket.
- 3) If the Millennium Blue Tray Liner is wider or longer than the interior of the basket, allow the edges of the Tray Liner to curve up and lie against the inside wall of the basket (see photo below).
- 4) When the Millennium Blue Tray Liner is smaller than the internal dimensions of the basket, position it centrally in the bottom of the basket
- 5) Place surgical instruments directly on top of the Millennium Blue Tray Liner.

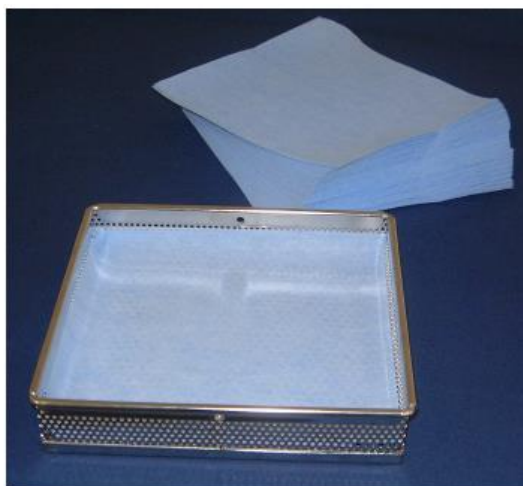

### Blue Heavy Tray Liners:

Blue Heavy Liner sheets can be used inside a tray or more commonly used under a tray to protect sterilisation wraps from internal damage and subsequent handling.