

Name: CONTAINER LOCKS

Product Codes: LOCK3030BLUE, LOCK3030BLUE1000, LOCKGRN605MIND

Name of Manufacturer



Intended Use

Container Locks are intended to be used to seal closed sterilization container with a tamper evident lock to insure container has not been open since sterilization.

Container Locks can also be affixed to any type of sterilization tray, the temperature indicator dot being used to indicate that the tray has been steam processed

Container Locks are intended to be used by professional healthcare personnel working in sterile services (or similar) departments.

Sterilisation Compatibility

Container Locks are compatible with steam sterilisation processes only

Single Use

Container Locks are single-use products. Once the temperature indicator lock has changed colour, do not re-use.

Storage Conditions

Storage best between 15-30°C and <60% Relative Humidity, preferably in a dry, dark cabinet.

Precautions

Container Locks should be stored away from any sources of ignition.


Disposal

Unused Container Locks may be disposed of to the plastic waste stream.

Used Container Locks should be disposed of by considering possible microbial contamination. Users should reference their own procedures in this matter.

How to use Key Surgical Container locks

- 1) Pass arrow shackle through opening of item to be sealed
- 2) Firmly push arrow shackle into locking device until it engages.
- 3) Pull on seal to ensure that it has been properly locked:

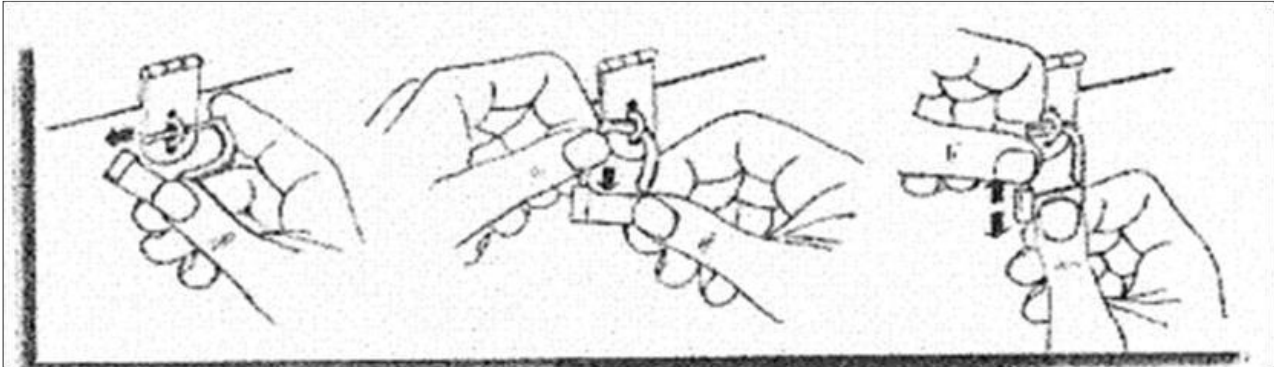

- 4) To remove, hook forefinger under locking barrel and pull laterally away from the slot.
Do not pull downward.
- 5) The reactive indicator dot will turn black or brown in the presence of steam sterilization for a minimum of 121°C/2min