

Product Name: Instrument/Tray Locks

Product Codes: LOCK994G, LOCK994S, LOCK994R

Name of Manufacturer:



Intended Use

Instrument/Tray Locks are a range of of special locking tags designed for marking instruments, trays and containers.

The LOCK994 variants are printed with 'Good', 'Repair' and 'Sharpen', specifically for use with instruments, for circulating between the user, the CSSD and repair shops.

Instrument/Tray Locks are intended to be used by professional healthcare personnel working in sterile services (or similar) departments.

Sterilisation Compatibility

Instrument/Tray Locks are compatible with steam, ETO and plasma sterilisation processes.

Single Use

Instrument/Tray Locks are single-use products. Once the seal of the lock has been broken, do not re-use.

Storage Conditions

There are no special storage requirements for Instrument/Tray Locks

Precautions

Instrument/Tray Locks should be stored away from any sources of ignition.

Disposal

Unused Instrument/Tray Locks may be disposed of to the plastic waste stream.

Used Instrument/Tray Locks should be disposed of by taking into account possible microbial contamination. Users should reference their own procedures in this matter.

How to use Key Surgical Instrument/Tray Locks

- 1) Place the open “spiked” end of the lock through the slot/hole/space of the item to be sealed.
- 2) Gently push the spike into the locking barrel until it engages the locking teeth
- 3) Gently push the spike further into the barrel to ensure that it has been properly locked – check that it cannot be pulled out.
- 4) To remove, hook forefinger under locking barrel and pull laterally away from the slot. Do not pull downward.

