

Magnetic Drapes

INTENDED USE:

Key Surgical Magnetic Drapes are a sterile device intended to be used in a hands-free transfer of stainless steel surgical sharps and instruments during procedures.

PRECAUTIONS:

This device is single use only; reuse may lead to patient cross contamination and/or device failure. Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.

CAUTION:

Do not use product in MRI controlled environments. Product is magnetic and not MRI safe. Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed.

CONTRAINDICATION:

Caution should be used during procedures with materials which may be adversely affected by magnetic fields.

INSTRUCTIONS FOR USE:

1. Magnetic drapes are provided sterile. Remove drape from pouch and place on appropriate surface.
2. Place stainless steel surgical sharps and instruments on drape as needed for transfer during procedure.

STORAGE REQUIREMENTS:

Store at room temperature in dry conditions. Do not expose to direct sunlight.

DISPOSAL:

Dispose according to facility protocol. Do not reuse.

 MANUFACTURER: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 USA
(US) PH: 800.541.7995 KEYSURGICAL.COM
(UK) TEL: +44(0)1628 810626 KEYSURGICAL.CO.UK
(EU) TEL: +49 4563 905900 KEYSURGICAL.DE

LOT

LOT
NUMBER

REF

CATALOG
NUMBER



CONSULT
INSTRUCTIONS
FOR USE



CAUTION



USE-BY DATE



DO NOT RESTERILIZE



DO NOT RE-USE

STERILE EO
STERILIZED USING ETHYLENE OXIDE



DO NOT USE IF PACKAGE
IS DAMAGED