## Vascular Loops

### EN - INTENDED USE:

Key Surgical® Vascular Loops are a disposable, single-use medical device intended for transient term (less than 60 minutes) use in surgical procedures requiring retraction, identification, or occlusion of peripheral vessels, nerves, ureters, or tendons. The Vascular loops are not intended for use in direct contact with the central nervous system.

### PRECAUTIONS:

This device is single use only; reuse may lead to patient cross contamination and/or device failure. Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.

### CAUTION:

Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed.

### CONTRAINDICATIONS:

There are no known contraindications associated with the Vascular Loops.

### INSTRUCTIONS FOR USE:

1. Remove vascular loops from pouch.
2. Remove and discard clip.
3. Vascular loops may be single- or double-looped and clamped at the site or may be clamped to vascular surgical drapes.

### DISPOSAL:

Upon completion of procedure, dispose of both used and unused loops according to facility protocol. Do not reuse.

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**MANUFACTURER: KEY SURGICAL LLC**