## Instrument Protectors: Paper / Plastic

### EN - INTENDED USE:

Key Surgical® Instrument Protectors are designed to protect the tips and edges of surgical instruments when not in use and during sterilization.

### PRECAUTION:

Single use only; reuse may lead to patient cross contamination.

### CONTRAINDICATIONS:

No known contraindications and/or adverse effects.

### INSTRUCTIONS FOR USE:

1. Does not need to be cleaned or disinfected before use.
2. Insert the sharp edge or point of the instrument through the paper bow and make sure the instrument is pushed all the way down.
3. Push the instrument down until under the edge of the laminate.
4. Single use and can be sterilized only once.

### STERILIZATION:

Compatible for sterilization according to the following parameters:

Steam Sterilization		
Cycle Type	Temperature	Minimum Time – Full Cycle
Pre-Vacuum	134° C (273° F)	3 minutes
Pre-Vacuum	134° C (273° F)	5 minutes
Pre-Vacuum	134° C (273° F)	18 minutes
Gravity	121° C (250° F)	15 minutes

It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

### STORAGE:

There are no special storage requirements.

### DISPOSAL:

Dispose after use according to facility protocol. Do not reuse.

### WARNING:

Serious incidents that have occurred in relation to this medical device should be reported to the manufacturer and competent authority in the country where the incident occurred.

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KEY SURGICAL LLC
UK REP MDSS UK RP LTD