Needle Counter without Blade Remover

EN - INTENDED USE:

Key Surgical® Needle Counters are a single use, sterile device designed to safely facilitate correct count and disposal of used blades and needles during surgical procedures.

PRECAUTIONS:

This device is single use only; reuse may lead to patient cross contamination and/or device failure.
Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.

CAUTION:

Do not use product in MRI controlled environments. Product is magnetic and not MRI safe.
USA Federal law restricts this device for sale by or on the order of a physician.
Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed.

CONTRAINDICATIONS:

Caution should be used during procedures with materials which may be adversely affected by magnetic fields.

INSTRUCTIONS FOR USE:

1. Remove adhesive backing on needle counter.
2. Open needle counter and place on flat surface.
3. Place used needles into container during procedure.
4. Close needle counter once container is at capacity with needles and/or procedure is ended.

DISPOSAL:

Dispose of needle counter in compliance with facility guidelines for disposing of used sharps.

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KEY SURGICAL LLC