

Name: Foam Cubes and Sheets

Product Codes: FOB004 –FOB999, FOS001-FOS999

Name of Manufacturer:



Intended Use:

Foam cubes and sheets are intended to be used for a variety of purposes within a healthcare environment.

Precautions:

Foam Cubes and sheets should be stored away from any sources of ignition.

Instructions for Use:

Foam cubes and sheets are intended for a variety of purposes and should be used in accordance with the facility's internal requirements and procedures.

Foam products, particularly sheets, can be used as an accessory to the sterilisation of medical instruments. Examples include tray liners and aids to positioning and protecting medical instruments during sterilisation.

Where the user requires the foam to be sterile prior to use, sterilise at 134-137°C for at least 3.5 minutes or more minutes. Ensure that the foam is not compressed during sterilisation. Foam cubes and sheets are not suitable for sterilisation by other processes.

Single use

Foam cubes and sheets are single use only.

Storage Conditions

Foam cubes and sheets should be stored in closed containers away from sources of light (to prevent yellowing). Foam cubes and sheets should not be exposed to temperatures exceeding 40 degrees°C and should be kept dry.

Disposal

Unused foam cubes and sheets should be disposed of according to local practice for foam products.

They should not be recycled in the plastic recycling stream

Used foam cubes and sheets should be disposed of by considering possible microbial contamination.

Users should reference their own procedures in this matter.