Electro Lube®
Electrosurgical Anti-Stick Solution

EN - INTENDED USE:

Electro Lube is a single patient use device that is intended to be used on electrosurgical electrodes to reduce sticking.

CONTRAINDICATIONS:

There are no known contraindications and/or adverse effects.

PRECAUTIONS:

- Contains: Lecithin (Soy). Soy is a potential allergen, which should be considered in treatment plans for individuals with known soy sensitivities.
- This device is single use only; reuse may lead to patient cross contamination.
- Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.

CAUTION:

- When using this device with other devices, follow all the instructions and safety warnings that come with that device.
- Visually inspect for breaches of packaging integrity prior to use. Do not use if opened, damaged or breached as sterility cannot be guaranteed.
- USA Federal law restricts this device to sale by or on the order of a physician.

INSTRUCTIONS FOR USE:

1. Dispense pouch contents onto sterile field.
2. Remove paper backing from foam pad and place foam pad onto sterile drape. Note: The foam pad includes a barium filament for radiopacity.
3. Remove cap and apply Electro Lube onto foam pad or dip electrode tip into bottle.
4. Wipe electrode tip on foam pad to leave a thin film.
5. Repeat as needed.

DISPOSAL:

Electro Lube should be disposed of after use; in accordance with facility standard practices.

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