




HELICOBACTER PYLORI TEST

The Clo Test is a convenient and compact test to provide you with the proof you need to clearly see the degree of infection of the Helicobacter Pylori in the digestive tract, with colour coded results. A ready to go, user friendly test with **no need for solution**.



M203

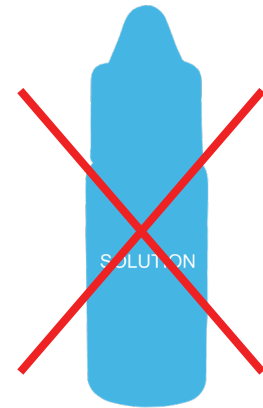

STEP 1

Take two or three biopsies with forceps.

STEP 2

Carefully peel open the test device without removing the film totally until it shows the yellow dot.

STEP 3


Place the tissue sample onto the yellow dot.

STEP 4

Close the peel off film and push firmly for one to two seconds to get the utmost contact.

STEP 5

Within 1 - 15 minutes the colour will change to show you a clear result.

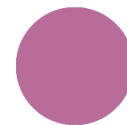

Positive for
contamination

Negative for
contamination

