

**Name: Non-Slip Foam Tray Pads**

**Product Codes: FMP001 – FMP999 inclusive**

**Name of Manufacturer: Clinipak Ltd**

**Address: Beech House  
Knaves Beech Business Centre  
Loudwater  
High Wycombe  
Bucks  
HP10 9SD**

**Tel: +44 (0)1628 810626**

**Fax: +44 (0)1628 810760**

**Website: [clinipak.co.uk](http://clinipak.co.uk)**

#### **Intended Use**

Non-Slip Foam Tray Pads are 3mm thick white foam sheets which are intended to be used as a liner for sterilisation trays which reduce movement of medical instruments during sterilisation.

Non-Slip Foam Tray Pads are intended to be used by professional healthcare personnel working in sterile services (or similar) departments.

#### **Sterilisation Compatibility**

Non-slip foam tray pads can be sterilised by steam or ethylene oxide.

Non-slip foam tray pads are not suitable for plasma sterilisation.

#### **Single Use**

Non-slip foam tray pads are single use only.

#### **Storage Conditions**

It is recommended that Non-Slip Foam Tray Pads are stored under the conditions stated on the label on the box. If this is not possible, they should be prevented from getting wet or from being exposed to high temperatures (>40degC).

#### **Precautions**

Non-Slip Foam Tray Pads should be stored away from any sources of ignition.

Non-Slip Foam Tray Pads should be stored in closed containers away from sources of light (to prevent yellowing)

#### **Disposal**

Unused Non-Slip Foam Tray Pads are not recyclable and should be disposed of according to local practice.

Used Non-Slip Foam Tray Pads should be disposed of by considering possible microbial contamination.

Users should reference their own procedures in this matter.

## How to use Non-Slip Foam Tray Pads

Non-Slip Foam Tray Pads are best suited for use with small or delicate surgical instruments.

- 1) Choose the most appropriate size for the basket to be used.
- 2) Place a Non-Slip Foam Tray Pad in the bottom of the basket.
- 3) If the Non-Slip Foam Tray Pad is wider or longer than the interior of the basket, allow the edges of the Non-Slip Foam Tray Pad to curve up and lie against the inside wall of the basket (see photo below).
- 4) If preferred, the Non-Slip Foam Tray Pad can be trimmed to a required size with a sharp knife or scissors. Ensure that any particles of foam produced by the cutting process are removed.
- 5) Place the instruments on top of the Non-Slip Foam Tray Pad.

