

**Name: Split Rings**

**Product Code: RIN001**

**Name of Manufacturer: Clinipak Ltd**

**Address: Beech House  
Knaves Beech Business Centre  
Loudwater  
High Wycombe  
Bucks  
HP10 9SD**

**Tel: +44 (0)1628 810626**

**Fax: +44 (0)1628 810760**

**Website: [clinipak.co.uk](http://clinipak.co.uk)**

### **Intended Use**

Split Rings are designed to be used for attaching Tray Tags and similar items to sterilisation trays and baskets.

Split Rings are intended to be used by professional healthcare personnel working in sterile services (or similar) departments.

### **Sterilisation compatibility**

Clinipak Rings are suitable for use in steam (up to 137°C), plasma and ethylene oxide sterilisation processes. They will withstand multiple sterilisation cycles - in particular at least 100 steam sterilisation cycles.

### **Storage Conditions**

There are no special storage condition requirements.

### **Precautions**

To avoid and chance of damage to fingers or nails, users are advised to use a device specifically designed for opening split rings. These are widely available from engineering supplies companies.

### **Disposal**

Split Rings are manufactured from stainless steel and is recyclable. If this option is not available, split rings can be disposed of through landfill.

## How to use Split Rings:

- 1) Open up the split ring either manually or with a device specifically for opening split rings.
- 2) Lever one of the two ends open as far as is required.
- 3) Slide the Tray Tag (or any other object that you are attaching) onto the open end using one of the 3 holes.
- 4) Push the Tray Tag around the Split Ring so that the Tray Tag dangles freely.
- 5) Using steps 1-4, attach the Split Ring to a basket or tray, using an appropriate hole or opening in the basket or tray.