

Name: Crepe Sterilisation Paper

**Product Code: WHCREPExxxx,
GRNCREPExxxx
WH/GCREPExxxx**

Name of Manufacturer: Clinipak Ltd
Address: Beech House
Knaves Beech Business Centre
Loudwater
High Wycombe
Bucks
HP10 9SD

Tel: +44 (0)1628 810626
Fax: +44 (0)1628 810760
Website: clinipak.co.uk

Intended Use

The intended use of Crepe Sterilisation Paper is as a microbial barrier sterilisation paper for use in sterile barrier packaging systems which are designed for the sterilisation of surgical instruments by steam, ethylene oxide or gamma radiation methods. It is used in combination with a second sterilisation wrap. The second sterilisation wrap can be another sheet of crepe sterilisation wrap or a different type of sterilisation wrap such as a nonwoven wrap.

Crepe Sterilisation Paper is intended to be used by professional healthcare personnel working in sterile services (or similar) departments.

Single Use

Crepe Sterilisation Paper is single use only.

Sterilisation Compatibility

Crepe Sterilisation Paper is compatible to steam, Ethylene Oxide, Formaldehyde and Radiation sterilisation techniques.

Storage Conditions

It is recommended that Crepe Sterilisation Paper is stored under the conditions stated on the label of the box. If this is not possible, it should be prevented from getting wet or from being exposed to high temperatures (>40degC).

Precautions

Crepe Sterilisation Paper should be stored away from any sources of ignition.
Like any paper sheet, crepe paper should be handled in such a way as to prevent cuts.

Disposal

Unused Crepe Sterilisation paper may be disposed of in the same way that the user normally disposes of paper. It is suitable for recycling in the waste paper stream.


Used Crepe Sterilisation Paper should be disposed of by considering possible microbial contamination. The user should reference their own procedures in this matter.



Wrapping with Crepe Sterilisation Paper

- 1) White and Green Crepe Sterilisation Paper can be used in the same way as any other standard Tray Wrap.
- 2) Crepe sterilization wrap should always be used in combination with at least one other tray wrap. When used with a non-woven or SMS-type Tray Wrap, the Crepe Sterilisation Wrap should be used as the inner wrap.
- 3) Any of the tray wrapping techniques can be used in either a sequential or simultaneous wrapping procedure.



Below are examples of 2 well-known wrapping techniques - **Parcel Fold and Envelope fold**. The examples use Green Crepe Sterilisation Paper as the inner wrap and an unspecified blue tray wrap as the outer wrap.

Parcel Fold

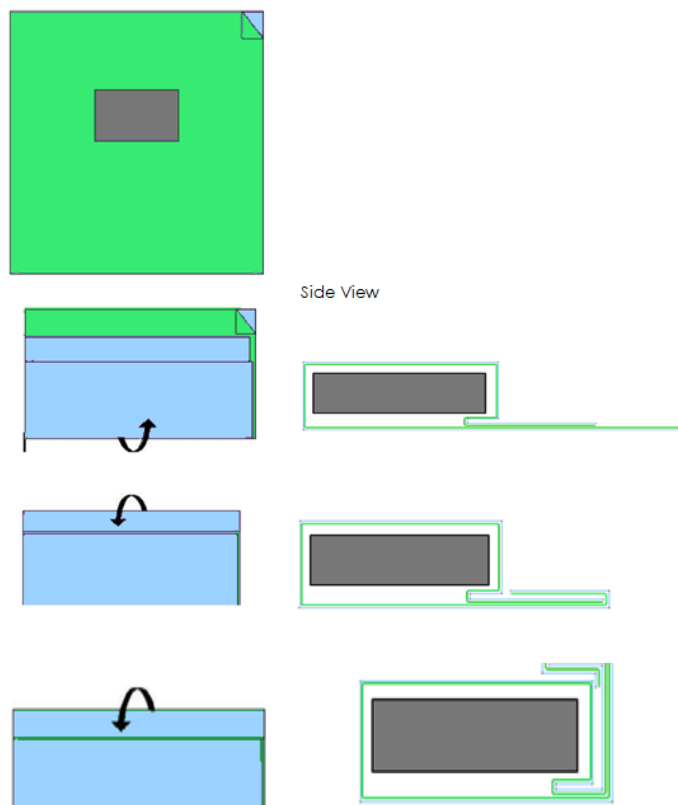

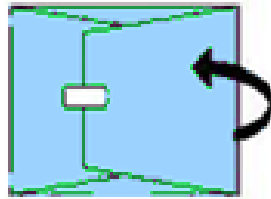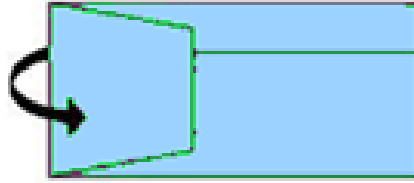

Envelope Fold

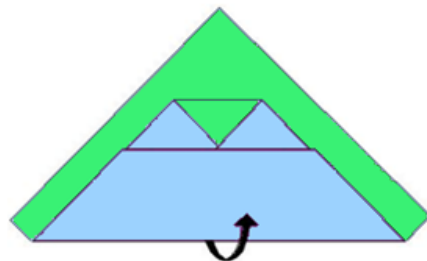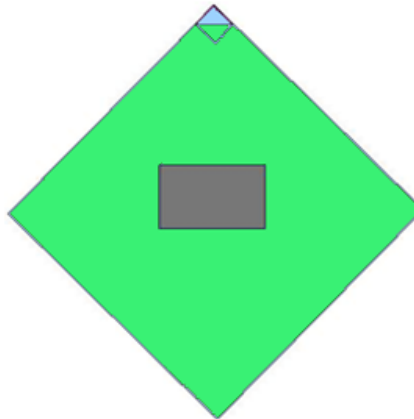

Side View



