

**Name: Paper Point and Needle Protectors**

**Product Codes: PPT001-PPT006**

**Name of Manufacturer: Clinipak Ltd**

**Address: Beech House**

**Knaves Beech Business Centre**

**Loudwater**

**High Wycombe**

**Bucks**

**HP10 9SD**

**Tel: +44 (0)1628 810626**

**Fax: +44 (0)1628 810760**

**Website: [clinipak.co.uk](http://clinipak.co.uk)**

### **Intended Use**

Paper Point and Needle Protectors are simple-to-assemble paper-based products which are designed for protecting chisels and other sharp instruments during sterilisation and during transport and storage. The Needle Protectors variant is designed for use in protecting fine sharp instruments such as Verres Needles, Silver Probes, Nettleship Dilators and Aspirating Needles.

Paper Point and Needle Protectors are intended to be used by professional healthcare personnel working in sterile services (or similar) departments.

### **Sterilisation Compatibility**

Paper Point and Needle Protectors are suitable for use in steam (up to 137°C) and ethylene oxide sterilisation processes. They are not suitable for use in plasma sterilisation processes.

### **Single Use**

Paper Point and Needle Protectors are single use only.

### **Storage Conditions**

It is recommended that Paper Point and Needle Protectors are stored under the conditions stated on the label of the box. If this is not possible, they should be prevented from getting wet or from being exposed to high temperatures (>40degC).

### **Precautions**

Paper Point and Needle Protectors should be stored away from any sources of ignition.

Like any paper product, Paper Point Protectors should be handled in such a way as to prevent cuts.





### **Disposal**

Unused Paper Point and Needle Protectors may be disposed of in the same way that the user normally disposes of paper. It is suitable for recycling in the waste paper stream.

Used Paper Point and Needle Protectors should be disposed of by considering possible microbial contamination. Users should reference their own procedures in this matter.

**How to use Paper Point and Needle Protectors:**

**1 Paper Point Protectors – PPT001, PPT002, PPT003, PPT004**

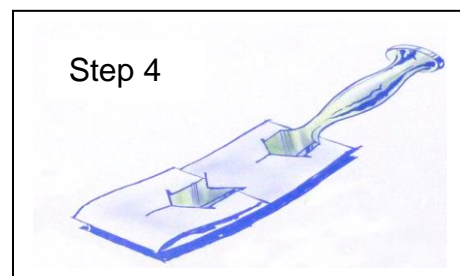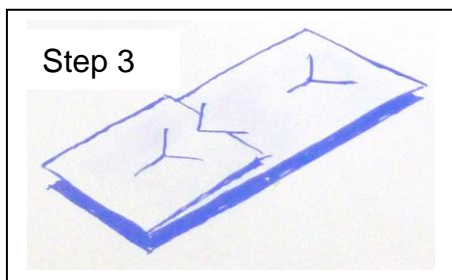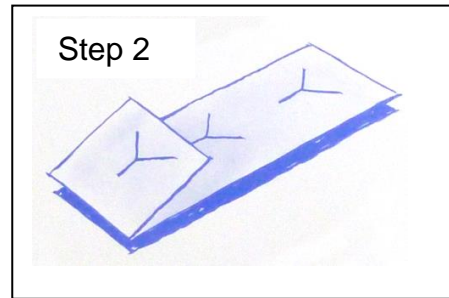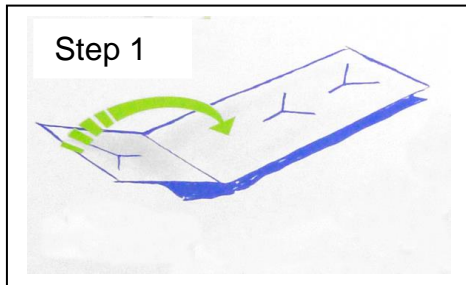

**2 Paper Needle Protectors – PPT005, PPT006**

