

Name: Absorbent Tray Liners

Product Codes: TLCxxx, TLCxxxx, CBLxxx, LINHxxx

Name of Manufacturer: Clinipak Ltd

Address: Beech House

Knaves Beech Business Centre

Loudwater

High Wycombe

Bucks

HP10 9SD

Tel: +44 (0)1628 810626

Fax: +44 (0)1628 810760

Website: clinipak.co.uk

Intended Uses

Absorbent Tray Liners can be used as:

- 1) A highly absorbent sheet tray Liner of wrapped instrument sets to prevent condensation.
- 2) A highly absorbent Tray Liner used underneath a tray to protect sterilization wrap from damage by tray corners in addition to the prevention of condensation.

Absorbent Tray Liners are intended to be used by professional healthcare personnel working in sterile services (or similar) departments and are single-use only.

Sterilisation Compatibility

Absorbent Tray Liners are suitable for use in steam (up to 137°C) and ethylene oxide sterilisation processes. They are not suitable for use in plasma sterilisation processes.

Single Use

Absorbent Tray Liners are single use only.

Storage Conditions

It is recommended that Absorbent Tray Liners are stored under the conditions stated on the label of the box. If this is not possible, they should be prevented from getting wet or from being exposed to high temperatures (>40degC).

Precautions

Absorbent Tray Liners should be stored away from any sources of ignition.

Like any paper product, Absorbent Tray Liners should be handled in such a way as to prevent cuts.


Disposal

Unused Absorbent Tray Liners may be disposed of in the same way that the user normally disposes of paper. It is suitable for recycling in the waste paper stream.

Used Absorbent Tray Liners should be disposed of by considering possible microbial contamination. Users should reference their own procedures in this matter.

Instructions for use**Tray Liner Sheets**

- 1) Choose the most appropriate size for the basket to be used.
- 2) Place the Absorbent Tray Liner in the bottom of the basket.
- 3) If the Absorbent Tray Liner is wider or longer than the interior of the basket, allow the edges of the Tray Liner to curve up and lie against the inside wall of the basket (see photo below).
- 4) When the Absorbent Tray Liner is smaller than the internal dimensions of the basket, position it centrally in the bottom of the basket.
- 5) Place surgical instruments directly on top of the Absorbent Tray Liner.

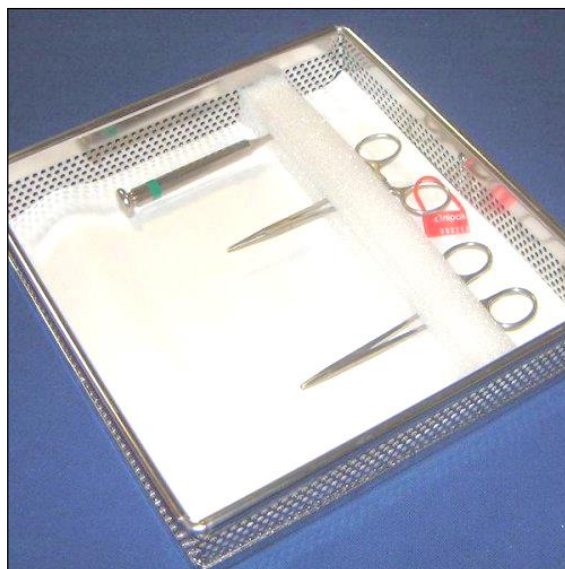

- 6) In an alternative arrangement, The Absorbent Tray Liner can be placed underneath the basket to provide additional protection to the sterilisation wrap against damage from the basket.

Tray Liner Rolls

Tray Liner rolls are used in the same way as the Tray Liner Sheets. The user should cut off the required length to fit the tray size

Cut Corner Tray Liners

Cut Corner Tray liners are intended to be used inside a sterilisation tray.

Fold the edges of the Cut Corner Tray Liners and then place in the bottom of the tray, allowing the edges to fit against the tray edges.

Heavy Liners

Heavy Liners are used under trays to not only protect inner wraps from the sharp corners or feet on trays and baskets, but to absorb condensate during sterilisation.

These sheets are manufactured from a heavier medical paper than standard tray liners and can be used as a shelf liner for added protection from rough handling and wet packs.