

**Name:** Choice Dry Bag Blue and Choice Dry Bag Green Bags

**Product Code:** CDBAGxxxB, CDBAGxxxG

**Name of Manufacturer:** Clinipak Ltd

**Address:** Beech House

Knave's Beech Business Centre

Loudwater

High Wycombe

Bucks

HP10 9SD

**Tel:** +44 (0)1628 810626

**Fax:** +44 (0)1628 810760

**Website:** [clinipak.co.uk](http://clinipak.co.uk)

### Intended Use

Clinipak Choice Dry Bag Blue and Choice Dry Bag Green Bags are single use, nonwoven, autoclavable bags which are designed to be used as outer sterilisation wraps in sterile barrier systems.

Choice Dry Bag Blue and Choice Dry Bag Green Bags are intended to be used by professional healthcare personnel working in sterile services (or similar) departments.

### Single Use

Choice Dry Bag Blue and Choice Dry Bag Green Bags are single use only.

### Sterilisation Compatibility

Choice Dry Bag Blue and Choice Dry Bag Green Bags are compatible only with steam and ETO sterilization processes.

### Storage Conditions

It is recommended that Choice Dry Bag Blue and Choice Dry Bag Green Bags are stored at 10-35°C, 20-80%RH. If this is not possible, it should be prevented from getting wet or from being exposed to high temperatures (>40degC).

### Precautions

Choice Dry Bag Blue and Choice Dry Bag Green Bags should be stored away from any sources of ignition.

Choice Dry Bag Blue and Choice Dry Bag Green Bags should not be used in H2O2/Plasma sterilization processes


## Disposal

Unused Choice Dry Bag Blue and Choice Dry Bag Green Bags can be disposed of in the plastic recycling stream.

Used Clinipak Cover Bags and Transportation Cover Dry Bags should be disposed of by taking into account possible microbial contamination. The user should reference their own procedures in this matter.

## How to Use Clinipak Cover Bags and Transportation Cover Dry Bags


- 1) Wrap the basket or tray containing the medical devices to be sterilized in inner barrier sterilization Wrap
- 2) Place the Wrapped basket or tray inside the Bag:

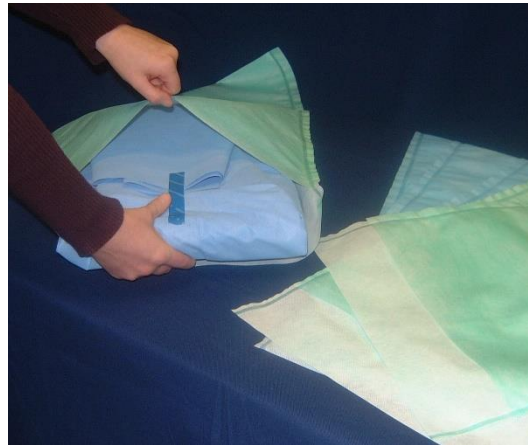


- 3) Fold over the lip of the bag:



- 4) Fold-over each edge of the Lip of the Bag:

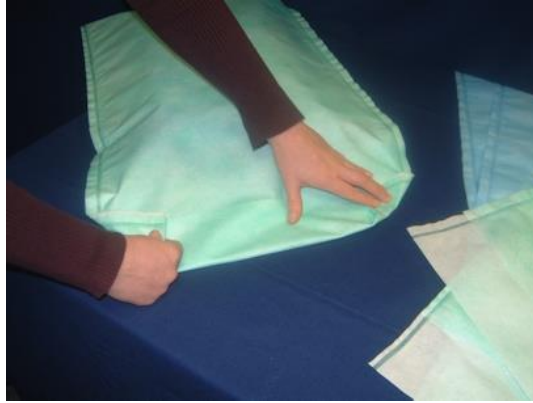

5) Fold-over the Lip 3 times:

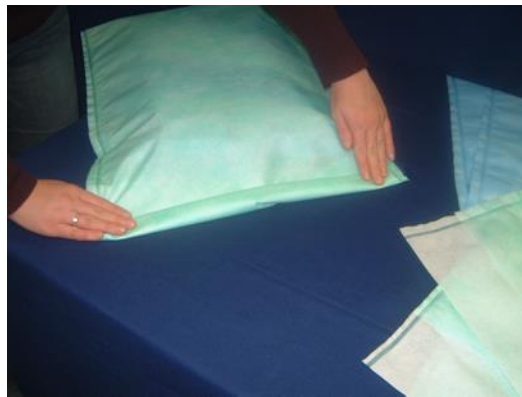

6) Place Autoclave tape over end:

