

Name : Orange Transportation Cover Bags

Product Code: TWOCOVERxxx,

Name of Manufacturer: Clinipak Ltd

Address: Beech House

Knave's Beech Business Centre

Loudwater

High Wycombe

Bucks

HP10 9SD

Tel: +44 (0)1628 810626

Fax: +44 (0)1628 810760

Website: clinipak.co.uk

Intended Use

Clinipak Orange Transportation Cover Bags are single use, nonwoven, autoclavable bags which are designed to be used as outer transportation wraps in sterile barrier systems.

Orange Transportation Cover Bags are intended to be used by professional healthcare personnel working in sterile services (or similar) departments.

Single Use

Orange Transportation Cover Bags are single use only.

Sterilisation Compatibility

Orange Transportation Cover Bags are compatible with steam, ETO and H₂O₂/Plasma sterilization processes.

Storage Conditions

It is recommended that Orange Transportation Cover Bags are stored at 10-35°C, 20-80%RH. If this is not possible, it should be prevented from getting wet or from being exposed to high temperatures (>40degC).

Precautions

Orange Transportation Cover Bags should be stored away from any sources of ignition.

Disposal

Orange Transportation Cover Bags can be disposed of in the plastic recycling stream.

Used Orange Transportation Cover Bags should be disposed of by considering possible microbial contamination. The user should reference their own procedures in this matter.

How to Use Clinipak Orange Transportation Cover Bags

- 1) Wrap the basket or tray containing the medical devices to be sterilized in 1 or more sterilization barrier wraps:
- 2) Place the Wrapped basket or tray inside the Bag:



- 3) Fold over the lip of the bag:



4) Fold-over each edge of the Lip of the Bag:



5) Fold-over the Lip 3 times:



6) Place Autoclave tape over end:

