

Name: Green STOPPER - Silicone Corner Edge Protector

Product Code: STOP-7100, STOP-7100P500

Name of Distributor: Clinipak Ltd
Address: Beech House
Knives Beech Business Centre
Loudwater
High Wycombe
Bucks
HP10 9SD

Tel: +44 (0)1628 810626

Fax: +44 (0)1628 810760

Website: clinipak.co.uk

Instructions for Use

1. Place surgical tray on top of desired sterile wrap.
2. Lift each bottom corner of surgical tray and slide the Green STOPPER under the foot or corner of tray.



3. Make sure the edge guards fit up against the surgical tray. The edge guards will not fit flush against the tray because of the dimples/studs inside of the edge guards. These dimples/studs allow for complete circulation of the sterilizing agent into the corners of the surgical tray.
4. Repeat step 2 above only invert the Green STOPPER to the top corners of the tray. After the Green STOPPER has been applied to all corners, continue to wrap the surgical tray as per hospital /Clinic/Surgery Centre protocol

Instructions for Cleaning

Point of use: Remove soil with disposable cloth or paper wipe.

Cleaning

1. Remove all visible soil and contaminants using soft bristle brush. The entire Green STOPPER should be immersed while cleaning to aid the in removal of contaminants and to reduce splashing of detergents on personnel.
2. Rinse thoroughly with clean water to remove all detergent.
3. If using Automated washer disinfectant process- After completion of automatic washer cycle, visually inspect the Green STOPPER for any remaining soil. All visible soil must be removed by hand cleaning, brushing, and additional automatic cycle prior to being used.

Washing and Sterilisation compatibility

Silicone Corners and Edge Protectors are compatible with all types of washing disinfecting cycles.

Silicone Corners and Edge Protectors are compatible with steam (up to 137°C), ethylene oxide and plasma sterilisation processes.