

**Name: Reliance Sterilisation Wrap**

**Product Code: RELGRNxxxx, RELBLUxxxx**

**Name of Manufacturer: Clinipak Ltd**

**Address: Beech House**

**Knives Beech Business Centre**

**Loudwater**

**High Wycombe**

**Bucks**

**HP10 9SD**

**Tel: +44 (0)1628 810626**

**Fax: +44 (0)1628 810760**

**Website: [clinipak.co.uk](http://clinipak.co.uk)**

### **Intended Use**

The intended use of Reliance Sterilisation Wrap is as a microbial barrier sterilisation wrap for use in sterile barrier packaging systems which are designed for the sterilisation of surgical instruments by steam, ethylene oxide or gamma radiation methods. It is used in combination with a second sterilisation wrap. The second sterilisation wrap can be another sheet of Reliance Sterilisation wrap or a different type of Sterilisation wrap such as a nonwoven wrap or a paper wrap.

Reliance Sterilisation Wrap is intended to be used by professional healthcare personnel working in sterile services (or similar) departments.

### **Single Use**

Reliance Sterilisation Wrap is single use only.

### **Storage Conditions**

It is recommended that Reliance Sterilisation Wrap is stored under the conditions stated on the label of the box. If this is not possible, it should be prevented from getting wet or from being exposed to high temperatures (>40°C).

### **Precautions**

Reliance Sterilisation Wrap should be stored away from any sources of ignition.

### **Disposal**

Unused Reliance Sterilisation wrap may be disposed of in the same way that the user normally disposes of wrap. It is suitable for recycling in the waste wrap stream.


Used Reliance Sterilisation Wrap should be disposed of by taking into account possible microbial contamination. The users should reference their own procedures in this matter.



**Wrapping with Reliance Sterilisation Wrap**

- 1) Blue and Green Reliance Sterilisation Wrap can be used in the same way as any other standard Tray Wrap.
- 2) Reliance sterilization wrap should always be used in combination with at least one other tray wrap; when used with a non-woven or SMS-type Tray Wrap, the Reliance Sterilisation Wrap can be used as either the inner or the outer wrap.
- 3) Any of the tray wrapping techniques can be used in either a sequential or simultaneous wrapping procedure.



Below are examples of 2 well-known wrapping techniques - **Parcel Fold and Envelope fold**  
 The examples use RELGRN Sterilisation Wrap as the inner wrap and the RELBLU Sterilisation Wrap as the outer wrap:

**Parcel Fold**

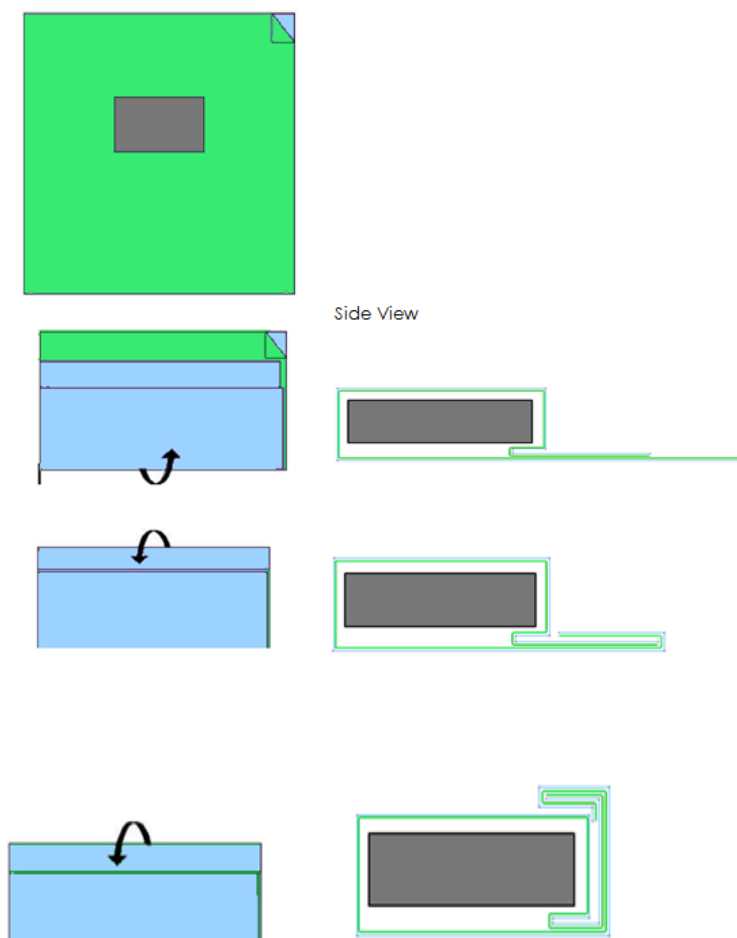

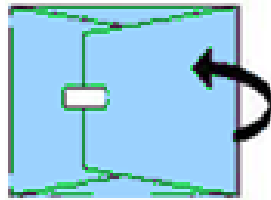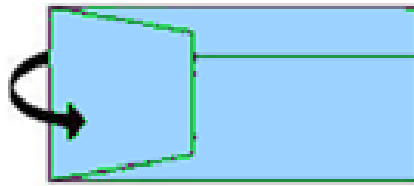

**Envelope Fold**

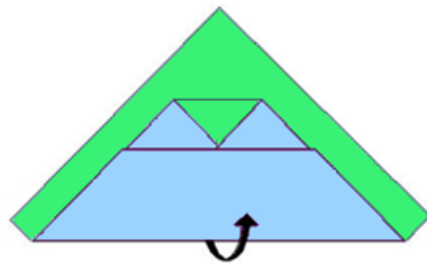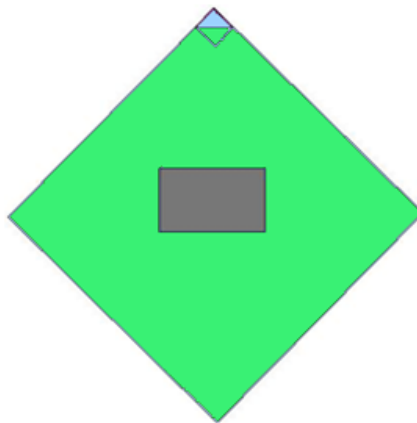

Side View



