EN **DIRECTIONS FOR USE**

The following information should be read before using this device.

INDICATIONS:

The Key Surgical Fog Free is the quickest and simplest method of preventing the formation of condensation on the distal lens tip of a laparoscope.

CONTRAINDICATIONS:

The device is not intended for use when minimally invasive techniques are contraindicated.

DEVICE DESCRIPTION:

The Key Surgical Fog Free is a 6ml bottle of solution with a mild alcohol content that immediately evaporates and prevents the formation of condensation on the distal lens tip of a laparoscope. Thereby, ensuring clear visualisation of the operative site throughout the procedure. A non-abrasive applicator sponge is included in order to assist with the removal of charred protein matter from the distal lens tip.

DIRECTIONS FOR USE:

- Using sterile technique, remove the bottle and applicator sponge from the packaging. To avoid impact with other instruments on the sterile table, do not flip the device into the sterile field.
- Place several drops of Fog Free on the distal lens tip of the laparoscope before the scope is placed into the abdominal cavity.
- This process may be repeated throughout the length of the procedure in order to ensure clear visualisation of the operative site.
- Where protein matter is charred onto the distal lens tip, a non-abrasive applicator sponge is provided to assist with its removal.

PRECAUTIONS/WARNINGS:

- This solution is designed for single use only and is not to be re-used or re-sterilized. Do not use if package is opened or damaged.
- Endoscopic procedures should be performed only by physicians with adequate training and knowledge of these procedures. In addition, medical literature should be consulted regarding techniques, hazards, contraindications and complications prior to the performance of these procedures.

SINGLE USE PRECAUTION:

These device(s) are designed and sold for single use only. Re-processing and, or, re-sterilisation is not permitted.

The effects of any unauthorised reprocessing or re sterilisation can result in the following complications:

1. Cross contamination due to ineffective re processing/re sterilisation.
2. Mechanical fatigue, and associated failure, due to the effects of the reprocessing / re sterilisation method.

After single patient use, the device is to be immediately disposed of as controlled medical waste.

HOW SUPPLIED:

The Key Surgical Fog Free is supplied sterile for single patient use.

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LABELLING AND SYMBOLS

M	h	g	H
Manufacturer	Catalogue number	Batch code	Use-by date
IK	i	Y	L
Sterilised using Gamma Radiation	Consult instructions for use	Caution	Do not use if package is damaged
D	B	p	W
Do not reuse	Do not re-sterilise	Keep dry	Keep away from sunlight
C1639	P		
CE Mark with notified body number	Authorised Representative		

DISTRIBUTED BY: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
(US) PH: 800.541.7995 KEYSURGICAL.COM
(UK) TEL: +44(0)1628 810626 KEYSURGICAL.CO.UK
(EU) TEL: +49 4363 905900 KEYSURGICAL.DE

M Purple Surgical International Ltd / Purple Surgical UK Ltd.
2 Chestnut House, Farm Close, Shenley, Herts WD7 9AD, England