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Vertrieb: Key Surgical GmbH

Produktname:

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Product name:

Silicone Counting Stripes

Please observe the processing instructions given in the following table, which relate to cleaning, disinfection and sterilization. While there are many processing options available, the correct choice depends on the material of the item at hand (i.e. its compatibility with the selected procedure). Note that the operator is solely responsible for the success of the selected procedure.

Preparation at location of use	
Wet or dry	Wet means the products are immersed in a non-fixating active cleaning disinfection solution immediately after the operation. Dry means the products are transferred to CSSD without any disinfectant or other additional fluids immediately after the operation.
Cleaning and disinfection	
Manual or mechanical reprocessing with/without ultrasound treatment	
Cleaning and disinfecting chemicals and temperatures	
Acidic / neutral / alkaline with/without tenside additives, chemically at max. 60°C / 140°F or with VE water thermally at max. 93°C / 199°F	It is assumed that commercially available products approved for the intended application are used and that the user complies with recommended concentrations, exposure times and temperatures. It must be guaranteed also that no residues remain on the products. Fully demineralized water must be used for the final rinse.
Drying	
Drying at max. 100°C / 212°F	
Checks, maintenance/care and inspection	
Inspection for usability and surface integrity	
Sterilization	
A choice of one or several processes: Validated steam sterilisation process with 134 °C/273 °C program or validated steam sterilisation process with 121°C/250 °C program.	Sterilisation and hold times are subject to national regulations and guidelines and, as such, cannot be stipulated in general terms. The operator bears responsibility for ensuring that reprocessing and sterilisation, as actually conducted with the equipment, materials and personnel used for reprocessing and sterilisation, achieve the required results. Validation and routine monitoring of the process are needed to that effect.
Alternative sterilisation processes*	*Note on alternative sterilisation processes: Steam sterilisation has become established worldwide as a very safe and reliable sterilisation method and is thus the method of choice for temperature- and humidity-resistant sterile supplies. Here reference is generally made to steam sterilisation using a validated steam sterilisation process (see also DIN EN ISO 17665). There is therefore no need to use alternative sterilisation processes, e.g. formaldehyde or ethylene oxide, for steam-tolerant products. However, each operator of a sterilisation unit is free to have sterilisation validation conducted using an alternative sterilisation process.
Storage	
Requirements	The silicone counting strips must be stored at room temperature. Do not expose to direct sunlight.
Other important notes	
<i>The processing information given herein cannot replace detailed process descriptions because it is not possible to give a detailed account of the numerous processing methods used all over the world. All information is given without guarantee. Please note: The silicone counting stripes are no medical devices.</i>	
Contact:	Key Surgical GmbH, Zum Windpark 1, 23738 Lensahn Tel. 04363/905900, Fax 04363/90590590

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Distributor: Key Surgical GmbH