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Hersteller: Key Surgical GmbH

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## Instructions for cleaning

Manufacturer: Key Surgical GmbH

**Product name:** Silicone sheet for packing table

Please observe the processing instructions given in the following table, which relate to cleaning and disinfection. Note that the operator is solely responsible for the success of the selected procedure.	
<b>Cleaning and Disinfection</b>	
Manual or machine processing <b>without</b> ultrasonic treatment.	
<b>Chemicals and temperatures to be used for cleaning and disinfecting</b>	
Acid/neutral/alkaline with/without addition of tensides, chemically at max. 60°C or with fully demineralized water, thermally at max. 93°C	It is assumed that commercially available products approved for the intended application are used and that the user complies with recommended concentrations, exposure times and temperatures. It must be guaranteed also that no residues remain on the products. Fully demineralized water must be used for the final rinse. - Don't use silicone oil -
<b>Drying</b>	
Max. 100 °C	
<b>Checks, maintenance/care and inspection</b>	
Inspection for usability	
<b>Storage</b>	
No special requirements	Please observe all basic rules and standard requirements
<b>Additional information</b>	
All information is given without guarantee.	
<b>Manufacturer:</b>	Key Surgical GmbH, Zum Windpark 1, 23738 Lensahn / Germany Phone +49 4363/905900, Fax +49 4363/90590590