Leonardo Wet-Set – REF 50083, 50078

EN INTENDED USE:

Leonardo Wet-Set for the transport storage of Bipolar forceps. Keeps the distal end moist and enhances the reprocessing of instruments.
Heat resistant up to 134 °C

CONTENTS:

Polypropylene tube, silicone lid

PRECAUTIONS:

N/A

CAUTION:

N/A

CONTRAINDICATIONS:

N/A

INSTRUCTIONS FOR USE:

Pre-Cleaning:

For pre-cleaning of the instruments please use water or a non-fixing disinfectant. Remove residues on the instrument's surface with a soft brush or wipe. Keep the instrument in a water-bath or rinse it with water afterwards.

1. The Leonardo Wet-Set is up for a one-time sterilization at 134 °C, 3.5 minutes before the use in the OR.
2. Take out the required amount of Leonardo Wet-Set tubes.
3. Fill cold demineralized water or a pH neutral cleaning agent which has been approved / recommended by the instruments manufacturer into the tubes.
4. Push the distal end of the instrument through the hole of the silicone lid.
5. Let the instrument soak during transport from the OR to the CSSD.
6. Pull the instrument out of the Leonardo Wet-Set.

DISPOSAL:

These products may pose a biological hazard after use. The product must therefore be handled and disposed of in accordance with current medical practice and local, state and national laws and regulations.

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