**REF 08006 Helix test - Formaldehyde indicator type 2**

EN

**INTENDED USE:**

For self-monitoring of the process parameters of steam/formaldehyde sterilization

**INSTRUCTIONS FOR USE:**

1. Unscrew the cap of the Helix test device.
2. Fold the chemical indicator strip in such way that the indicator is on the inside.

3. Insert the folded indicator strip with the closed end first in the cap of the Helix test device.

4. Insert the indicator strip completely.

5. Close Helix test device by screwing the cap firmly.

6. Place the Helix test device close the bottom of the sterilizer and sterilize it together with the load.

7. Indicator Evaluation:

- a. Open Helix test device and remove the indicator strip.  
Color Change of Formaldehyde Indicators for sterilization over 65 °C:

Before sterilization = beige

After correct sterilization: all indicator fields are evenly deep violet.

**Failed colour change:**

The fields on the indicator strip have different colors (in case of use in challenge device).

Possible Reason:

Inadequate air removal

The fields on the indicator strip have evenly light violet color.

Possible Reason:

Insufficient work of formaldehyde, e.g. due to overload.

**NOTE:**

Direct contact of the indicator with the sterilized material, e.g. by sticking, must be avoided, as incorrect reactions and contamination of the sterilized material can occur through metal contact, for example. The self-adhesive property of the indicator is only suitable for documentation after completion of sterilization. This is done by peeling off the indicator strip at the perforation above the FORM imprint.

**STORAGE:**

The indicator is sensitive to light. To ensure that the indicator is not affected by exposure to light or other environmental influences, it should be removed from its packaging only immediately prior to use. Sterilized indicators, too, must be stored while protecting them in a similar manner.

Storage Temperature: 5-30°C

Humidity: 30-60%

**DISPOSAL:**

There are no special disposal instructions.

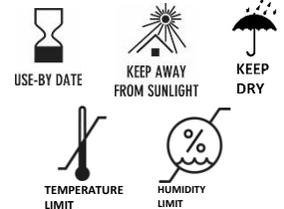
MANUFACTURER: KEY SURGICAL GMBH  
ZUM WINDPARK 1 / 23738 LENSahn, GERMANY  
TEL: +49 4363 905900 KEYSURGICAL.DE

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