Leonardo Wet-Set – REF 50055, 50058, 50065, 50066**EN INTENDED USE:**

Leonardo Wet-Set for the transport storage of EndoWrist®, laparoscopic or other lumen instruments and for Instruments for High Frequency Surgery. Keeps the distal end moist and enhances the reprocessing of instruments.
Heat resistant up to 134 °C

CONTENT:

Polypropylen tube, silicone lid

PRECAUTIONS:

This device is single use only; reuse may lead to patient cross contamination and/or device failure.
Wear gloves and/or proper protective equipment when washing, as prolonged contact with solution may cause skin and eye irritability.

CAUTION:

N/A

CONTRAINDICATIONS:

N/A

INSTRUCTIONS FOR USE:

1. The Leonardo Wet-Set is up for a one-time sterilization at 134 °C, 3.5 minutes before the use in the OR.
2. Take out the required amount of Leonardo Wet-Set tubes.
3. Fill cold demineralized water or a pH neutral cleaning agent which has been approved / recommended by the instruments manufacturer into the tubes.
4. Push the distal end of the instrument through the hole of the silicone lid and open the instrument tip, if possible.
5. When using with EndoWrist® instruments inject 15 ml of liquid into the primary flush port 1 and some liquid into the secondary flush port 2.
6. Let the instrument soak during transport from the OR to the CSSD.
7. Pull the instrument out of the Leonardo Wet-Set.
8. Dispose the Leonardo Wet-Set according to your waste disposal regulations.

DISPOSAL:

These products may pose a biological hazard after use. The product must therefore be handled and disposed of in accordance with current medical practice and local, state and national laws and regulations.

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