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Hersteller: Key Surgical GmbH

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Instructions for cleaning

Manufacturer: Key Surgical GmbH

Product name: Disposal box made of PP
REF 09811, 09813, 09824, 09844, 09895, 09893, 09896, 09897, 09896,
09894

Please observe the cleaning instructions in the following table. The instructions concern cleaning and disinfection. The success of cleaning and disinfection is the sole responsibility of the operator.

Cleaning and Disinfection	
Manual or machine processing (in RDT machine).	
Chemicals and temperatures to be used for cleaning and disinfecting	
Acid/neutral/alkaline with/without addition of tensides, chemically at max. 90°C or with fully demineralized water, thermally at max. 90°C	It is assumed that commercially available products approved for the intended application are used and that the user complies with recommended concentrations, exposure times and temperatures. It must be guaranteed also that no residues remain on the products. Fully demineralized water must be used for the final rinse.
Drying	
Max. 95 °C	
Control, maintenance and testing	
Visual inspection for usability.	
Storage	
No special requirements	Please observe all basic rules and standard requirements
Additional information	
All information is given without guarantee.	
Manufacturer	Key Surgical GmbH, Zum Windpark 1, 23738 Lensahn Tel. 04363/905900, Fax 04363/90590590