Sterilisation Mats

EN - INTENDED USE:

Key Surgical® Sterilisation Mats are intended to be placed under the wrapped sterilisation tray for organization and storage.

CONTRAINDICATIONS:

No known contraindications and/or adverse effects.

PRECAUTIONS:

Store away from any sources of ignition.

Single use only; reuse may lead to patient cross contamination.

INSTRUCTIONS FOR USE:

1. Does not need to be cleaned or disinfected before initial use
2. Choose a mat which is slightly larger than the underside of the tray or basket being wrapped.
3. Place the mat on a flat surface.
4. Place the mat on the sterilisation wrap in the place where you intend to put the basket.
5. Place the basket or tray on the mat.
6. When using DRYMAT or DRYMATGRN, ensure that the white side is uppermost, so it contacts the tray or basket.
7. Load basket or tray and wrap as normal.

STERILISATION:

Evaluated for sterilisation compatibility and found to be compatible with steam sterilisation. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

DISPOSAL:

Used: Dispose after use according to facility protocol. Do not reuse.

Unused: Can be disposed of in the plastic recycling stream.

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MANUFACTURER: KEY SURGICAL LLC