## Disposable Corner Protectors (Foam)

### EN - INTENDED USE:

Key Surgical® Tray Corner Protectors are intended for use with surgical trays to prevent the tearing of the sterilization wrap and to protect the corners of the trays from damage.

### CONTRAINDICATIONS:

No known contraindications and/or adverse effects.

### PRECAUTIONS:

Store away from any sources of ignition.  
Single use only; reuse may lead to patient cross contamination.

### INSTRUCTIONS FOR USE:

- Does not need to be cleaned or disinfected before use.
- Place the corner protectors on the top and bottom corners of the tray before wrapping with sterilization wrap.
- Wrap the tray according to standard wrapping procedures. Appropriate sterilant contact will occur on the surface of the corner protectors as well as the tray surface under the corner protectors.

### STERILIZATION:

Evaluated for sterilization compatibility and found to be compatible with the following parameters:

Steam Sterilization		
Cycle Type	Temperature	Minimum Time – Full Cycle
Pre-Vacuum	132° C (270° F)	4 minutes
Pre-Vacuum	134°C (273°F)	3.5 minutes
Gravity	121° C (250° F)	30 minutes
Gravity	121°C (250°C)	15 minutes

It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

### DISPOSAL:

Dispose after use according to facility protocol. Do not reuse.

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MANUFACTURER: KEY SURGICAL LLC