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Distributor: Key Surgical GmbH

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Reprocessing Instructions

Distributor: Key Surgical GmbH

Product name: Mesh sleeves made of stainless steel

The user is responsible for the success of the reprocessing cycle. Please take note of the details and regulations in the instruction manual as well as the legal obligations and standards.	
Preparation at location of use	
wet or dry	Wet means the products are immersed in a non-fixating active cleaning disinfection solution immediately after using. Dry means the products are transferred to CSSD without any disinfectant or other additional fluids immediately after using.
Cleaning and disinfection	
manual or mechanical reprocessing with or without ultrasound treatment	
Cleaning and disinfecting chemicals and temperatures	
acidic / neutral / alkaline with or without tenside additives, chemically at max. 60° C / 140° F or with demineralized water, thermally, at max. 93° C / 199° F	Assuming that the products used for cleaning and disinfection are commercially available and licensed for the respective application, and that the recommended concentrations, exposure times and temperatures are observed. It must be ensured that no residue remains on the products. Demineralized water should be used for final rinsing.
Drying	
drying at max. 120° C / 248° F	
Sterilization processes	
STEAM 134 °C / 273 °F / 5 min 134 °C / 273 °F / 18 min 121 °C / 250 °F / 20min.	Sterilization and holding times are subject to national regulations and guidelines and therefore cannot be set universally. It is the operator's responsibility to ensure that the intended results are actually achieved with the processing and sterilization methods, equipment, materials and personnel used. This, in turn, requires validation and routine monitoring of all relevant processes.
Other sterilization processes	
Alternative sterilization methods* formaldehyde (max. 65 °C / 149 °F), ethylene oxide (max. 65 °C / 149 °F), VH2O2 (max. 50°C / 122 °F)	*Notes on alternative sterilization methods: Steam sterilization has established itself worldwide as a very safe and highly reliable sterilization method and, therefore, represents the method of choice with regard to temperature- and humidity-insensitive goods. As a rule, steam sterilization in the form of a validated process is required (see also DIN EN ISO 17665-1). Therefore, it is not necessary to use alternative sterilization methods – such as low-temperature, formaldehyde, ethylene oxide or VH2O2 for steam-sterilizable products. Nonetheless, sterilizer operators are free to validate sterilization processes using alternative methods for sterilizing their products.
Storage	
no special requirements	
Other important notes	
<i>The reprocessing advice give here is no substitute for detailed process manuals as, in this document, we cannot deal with all details of the large variety of reprocessing processes that are used worldwide. No responsibility is taken for the correctness of the information given in this respect.</i>	
Contact:	Key Surgical GmbH, Zum Windpark 1, 23738 Lensahn Tel. 04363/905900, Fax 04363/90590590