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## Instruction for use

REF 26050, 26052  
Einmal-Biopsieventile (unsteril)  
Single-use Biopsy Valves (non-sterile)

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Training recommendations:  
Please read this manual carefully before first use.  
Keep this manual for future reference.

### 1.0 Allgemeines

#### General information

MANUFACTURER: KEY SURGICAL LLC  
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA  
PH: 800.541.7995 KEYSURGICAL.COM

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The biopsy valves are designed for use with the endoscopes described in section 5.0 and are intended to prevent the backflow of body fluids. The valves should not be used for purposes other than those described.

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The valves are for single use only.  
If faults are discovered before use, replace the valve with a new one. Check the valve for cracks, fissures or deformations, and replace if necessary.

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#### Connect the biopsy valve to the endoscope

TEDA-4114, Rev. D

Press the valve onto the biopsy channel opening until the valve clicks into place.  
NOTE: At low temperatures, the biopsy valves become tighter and may become heavier to connect.

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#### Supply a solution using a syringe

Insert the syringe firmly and hold it perpendicular to the valve. Press the syringe downwards to supply the solution.

**Caution:** Angled or incomplete insertion of the syringe may cause valve leakage.

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#### Aspirating liquids by syringe

Remove the valve from the biopsy channel opening and insert a syringe. Pull the syringe upwards to suck the fluid.

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**Caution:** Insert the accessories straight into the biopsy valve.  
Insert the accessories through the opening into the biopsy valve.  
Remove endo therapy accessories slowly after application.

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#### Removing the biopsy valve from the endoscope

Carefully pull the valve from the biopsy channel opening.

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REF 26050 Suitable for OLYMPUS & FUJINON GI ENDOSCOPE

REF 26052 Suitable for PENTAX GI ENDOSCOPE

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Store the biopsy valves dry at room temperature and do not expose them to direct sunlight.

TEDA-4114, Rev. D

**CAUTION:** Do not store the biopsy valves in the transport container of the endoscope. Use the transport container exclusively for the endoscope, as the permanent storage of the disposable valves in a closed, moist, non-ventilated container may result in an infection risk.

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