Endoscope Tray Cover – Non-sterile Abdeckhaubenset für Endoskopschalen – unsteril

EN INTENDED USE:

The Endoscope Tray Covers are intended to protect endoscopes by reducing the exposure to airborne contaminants and minimizing direct handling of the endoscope during transportation and storage.

PRECAUTIONS:

This device is single use only; reuse may lead to patient cross contamination and/or device failure.

CAUTION:

N/A

CONTRAINDICATIONS:

There are no known contraindications associated with endoscope tray covers.

INSTRUCTIONS FOR USE:

1. Remove the device components from the peel pouch.
2. Insert the endoscope tray into the clear bag and close the clear bag with the adhesive seal.
3. Unfold the tray liner and place it over the tray on the outside of the clear bag.
4. Place the clean endoscope in the tray on top of the tray liner.
5. For style 17215:
 - a. Secure the hood over the tray, covering the clean endoscope. Ensure that the "CLEAN" imprint faces outwards. Transport the covered endoscope to the point of use.
 - b. At point of use, remove the hood from the tray to access the endoscope. Do not discard the hood or remove the tray liner and clear bag from the tray.
 - c. Post-procedure, place the contaminated endoscope back in the tray on top of the tray liner. Secure the hood over the tray, covering the contaminated endoscope. Ensure that the "CONTAMINATED" imprint faces outwards. Transport the endoscope to the decontamination unit.
6. For style 17216:
 - a. Place the folded red hood into the tray with the clean endoscope.
 - b. Secure the green hood over the tray, covering the clean endoscope. Transport the covered endoscope to the point of use.
 - c. At point of use, remove the green hood from the tray to access the endoscope. Discard the green hood in normal waste. Do not remove the tray liner and clear bag from the tray.
 - d. Post-procedure, place the contaminated endoscope back in the tray on top of the tray liner. Secure the red hood over the tray, covering the contaminated endoscope. Transport the endoscope to your decontamination unit.
7. Discard the hood and remove the endoscope from the tray. Carefully invert the clear bag over the tray liner to remove the tray from the clear bag. Take care to contain any fluid inside the clear bag. The clear bag and tray liner can then be discarded per facility protocol for contaminated waste.

DISPOSAL:

There are no special disposal instructions.

 **MANUFACTURER: KEY SURGICAL LLC**
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM

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