EN – INTENDED USE:

ColourCurls® are created for the identification of medical instruments.

ColourCurls® mini	REF 09620 - 09632
ColourCurls® maxi	REF 09639 - 09651
ColourCurls® supermaxi	REF 09663 - 09675

PRECAUTION:

Protruding ends can damage gloves.
 Make absolutely sure that ColourCurls® are fitted as described.
 After using an instrument with ColourCurls®, make sure that ColourCurls® continue to remain fitted to the instrument.
 ColourCurls® are equipped with X-ray contrast medium.

PREPARATION:

ColourCurls® must be affixed only to cleaned instruments.

INSTRUCTIONS FOR USE:

 <p>1</p>	<p>The ColourCurls® are suitable for instruments with ring handles: ColourCurls® mini: Diameter up to approx. 3,5 mm ColourCurls® maxi: Diameter up to approx. 6 mm ColourCurls® supermaxi: Diameter up to approx. 7 mm</p>
 <p>2</p>	<p>Place ColourCurls® with the central winding on the instrument.</p>
 <p>3</p>	<p>Now wind both ends around the instrument until ColourCurls® in its entirety have enclosed the desired area of the instrument.</p>
 <p>4</p>	
 <p>5</p>	<p>ColourCurls® should be positioned on the instrument as illustrated in Figures 4 and 5. If the ends are protruding (bent), new ColourCurls® must be fitted.</p>

DISPOSAL:

There are no special disposal instructions.

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KEY SURGICAL LLC