**Enginol REF 02006  
Slide Way Oil REF 02005**

**EN - INTENDED USE:**

Enginol / slide way oil are used for the care and maintenance of the mobility in compressed air machines, motor systems and microtomes.

- Enginol: Special care oil for compressed air machines and sterilizable motor systems.
- Slide way oil: oil for microtomes

**CAUTION:**

May be fatal if swallowed and enters airways.

IF SWALLOWED: Immediately call a Poison Center / doctor.

Do NOT induce vomiting

Store locked up.

**INSTRUCTIONS FOR USE:**

1. Shake well before use.
2. Apply a few drops of Enginol/slideshow oil to the desired area.
3. Instruments can be sterilized according to the parameters listed here.

**STERILIZATION:**

Validated for sterilization according to the following parameters:

Cycle Type (Steam)	Temperature	Minimum Time – Full Cycle
Fractionated Pre-Vacuum	134° C (273° F)	3.5 minutes
Fractionated Pre-Vacuum	121° C (250° F)	15 minutes

**STORAGE:**

There are no special storage conditions

**DISPOSAL:**

For proper disposal please empty product completely and dispose after use according to facility protocol.

**WARNING:**

Serious incidents that have occurred in relation to this medical device should be reported to the manufacturer and competent authority in the country where the incident occurred.

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**KEY SURGICAL LLC**
**UK REP** MDSS UK RP LTD