TAPE-EX

EN - INTENDED USE:

TAPE-EX is designed to remove soil and residue from adhesive tapes and labels on medical instruments.

CAUTION:

Extremely flammable aerosol.

Pressurized container: May burst if heated.

Keep out of reach of children.

Keep away from heat, hot surfaces, sparks, open flames, and other ignition sources. No smoking.

Do not spray on an open flame or other ignition source.

Do not pierce or burn, even after use.

Avoid release to the environment.

Protect from sunlight.

Do not expose to temperatures exceeding 50°C/122°F.

Buildup of explosive mixtures possible without sufficient ventilation.

INSTRUCTIONS FOR USE:

1. Spray TAPE-EX onto a cloth and clean respective area.
 2. Soaking such spots with TAPE-EX is advisable for stubborn soils.
 3. Allow surface to dry before reapplying labels or instrument identification.
- Consult the instrument manufacturer's IFU for instrument reprocessing information.

STORAGE:

Store at +42°F (+5°C) to 86°F (+30°C).

DISPOSAL:

For proper disposal please empty product completely and dispose after use according to facility protocol.

WARNING:

Serious incidents that have occurred in relation to this medical device should be reported to the manufacturer and competent authority in the country where the incident occurred.

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KEY SURGICAL LLC
MDSS UK RP LTD