General Instrument Cleaning Brushes

EN - INTENDED USE:

Key Surgical® General Instrument Cleaning Brushes are intended to clean dirty instruments during the cleaning and decontamination process.

CONTRAINDICATIONS:

No known contraindications and/or adverse effects.

INSTRUCTIONS FOR USE:

1. Wear protective clothing such as an apron and rubber/plastic gloves. Eye protection is also recommended.
2. Perform the cleaning in a sink with the tap water running - preferentially use hot water.
3. The cleaning brushes can be used in combinations with liquid cleaners and surfactants.
4. Hold the cleaning brush in one hand, anchor the instrument / device to be cleaned with the other hand and scrub the required part of the instrument / device until the contamination is removed.
5. For bottle style cleaning brushes insert the brush into the opening of the instrument / device and scrub the inside of the instrument / device using an in-out motion of the cleaning brush until all the contaminant is removed.
6. Whilst cleaning instruments / devices continually wash the contaminant away down the sink.
7. Give the instrument / device a thorough wash with running water.
8. Avoid contaminant flicking out of the sink onto the user or nearby personnel.

CLEANING & DISINFECTION:

Clean brushes after each use and disinfect between cases or daily, at a minimum. Can be cleaned manually or using an automatic washer/disinfector.

INSPECTION:

Inspect between uses and replaced when worn, frayed, bent or otherwise damaged. Damaged brushes cannot be cleaned effectively. Worn or damaged bristles and handles are ineffective in cleaning, and may damage the device. Requires replacing if there are any signs of deterioration or loss of functionality

STERILIZATION:

NOTE: Product code BBR002 is NOT suitable for sterilization. This sterilization is intended for one time use of brushes in a sterile setting, and is not for brushes that have already been used.

New, unused brushes are provided non-sterile and have been validated for sterilization efficacy using gravity and pre-vacuum steam sterilization methods according to the following parameters:

Steam Sterilization		
Cycle Type	Temperature	Minimum Time –Full Cycle
Gravity	121° C (250° F)	30 minutes
Pre-Vacuum	132° C (270° F)	4 minutes
Pre-Vacuum	134° C (273° F)	18 minutes

STORAGE:

Store brushes in a manner that will reduce cross-contamination.

DISPOSAL:

Dispose according to facility protocol.

WARNING:

Serious incidents that have occurred in relation to this medical device should be reported to the manufacturer and competent authority in the country where the incident occurred.

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KEY SURGICAL LLC
UK REP MDSS UK RP LTD