Burr Cleaning Brushes

EN - INTENDED USE:

Key Surgical® Burr Cleaning Brushes are single use brushes specially designed for cleaning surgical instruments during a procedure to keep burrs, rasps, files and saws free of debris.

CONTRAINDICATIONS:

No known contraindications and/or adverse effects.

PRECAUTION:

Single use only; reuse may lead to patient cross contamination.

INSTRUCTIONS FOR USE:

- Does not need to be cleaned or disinfected prior to use but can be cleaned manually or using an automatic washer/disinfector if desired.
- After sterilization, move wrapped set to sterile storage area for storage or transport to point of use location.

STERILIZATION:

Provided non-sterile and have been validated for sterilization efficacy using gravity and pre-vacuum steam sterilization methods according to the following parameters:

Steam Sterilization		
Cycle Type	Temperature	Minimum Time – Full Cycle
Gravity	121° C (250° F)	30 minutes
Pre-Vacuum	132° C (270° F)	10 minutes
Pre-Vacuum	134° C (273° F)	18 minutes
Pre-Vacuum	134° C (273° F)	3.5 minutes

DISPOSAL:

Dispose according to facility protocol. Do not reuse.

WARNING:

Serious incidents that have occurred in relation to this medical device should be reported to the manufacturer and competent authority in the country where the incident occurred.

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KEY SURGICAL LLC
MDSS UK RP LTD