
## Ident Pearls

### EN – INTENDED USE:

Ident Pearls are created for the identification of medical instruments.

### DEFINITION: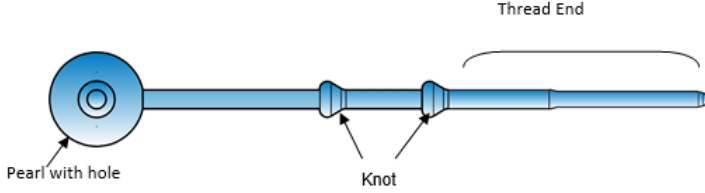

Ident Pearl mini	REF 09271 - 09281
Ident Pearl maxi	REF 09289 - 09299
Ident Pearl supermaxi	REF 09652 - 09662

### PRECAUTION:

Make sure that after using the instrument the Ident Pearl still fits perfectly. Ident Pearl is equipped with X-ray contrast medium.

### PREPARATION:

Ident Pearls must be affixed only to cleaned instruments.

### INSTRUCTIONS FOR USE:

<p>1</p>	<p>The Ident Pearls are suitable for instruments with ring handles: Ident Pearl mini: Diameter up to approx. 4 mm Ident Pearl maxi: Diameter up to approx. 8 mm Ident Pearl supermaxi: Diameter up to approx. 10-20 mm</p>
<p>2</p>	<p>Place the Ident Pearl around the instrument and run the thread end through the hole of the pearl.</p>
<p>3</p>	<p>Pull at the thread end until the knot completely passes through the pearl and adjusts itself on the other side. The pearl is secured in this way by a knot on both sides of the pearl.</p>
<p>4</p>	<p>Pull at the thread end until the knot completely passes through the pearl and adjusts itself on the other side. The pearl is secured in this way by a knot on both sides of the pearl.</p>
<p>5</p>	<p>For smaller instrument diameters, the second knot can be pulled completely through the pearl.</p>

Now cut off the oddment behind the knot.  
(Don't cut off the knot itself)  
Check that Ident Pearl fits tightly to the instrument.

### DISPOSAL:

There are no special disposal instructions.

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KEY SURGICAL LLC