## Lubrinol

### EN - INTENDED USE:

Maintenance oil for surgical instruments.

Property:

- high creeping effect
- lubricating quality for joints
- physiologically safe

### CAUTION:

Lubrinol is not to be used for optics.

Extremely flammable aerosol. Pressurised container: May burst if heated.

Harmful to aquatic life with long lasting effects.

Keep out of reach of children.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.

No smoking.

Do not spray on an open flame or other ignition source.

Do not pierce or burn, even after use.

Avoid release to the environment.

Protect from sunlight.

Do not expose to temperatures exceeding 50 °C/122 °F.

Buildup of explosive mixtures possible without sufficient ventilation.

### INSTRUCTIONS FOR USE:

1. Shake well before use.
2. Spray cleaned instruments at a distance of 8-12" (20-30cm).
3. Sterilize instruments according to parameters listed here

### STERILIZATION:

Validated for sterilization according to the following parameters:

Cycle Type (Steam)	Temperature	Minimum Time – Full Cycle
Pre-Vacuum	134° C (273° F)	3.5 minutes

### STORAGE:

Store at +42°F (+5°C) to 86° F (+30°C).

### DISPOSAL:

For proper disposal please empty product completely and dispose after use according to facility protocol.

### WARNING:

Serious incidents that have occurred in relation to this medical device should be reported to the manufacturer and competent authority in the country where the incident occurred.

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KEY SURGICAL LLC
MDSS UK RP LTD