



Adhesive Remover

INTENDED USE:

Adhesive remover is designed to dissolve tape residue on instruments, trays, basins, or other surfaces.

PRECAUTION:

Use with adequate ventilation. In case of contact with eyes or skin, wash affected area thoroughly with water. If irritation persists, seek medical attention. Keep out of reach of children. Avoid contact with food. If swallowed, DO NOT induce vomiting and seek medical attention immediately.

Reference the Adhesive Remover Safety Data Sheet (SDS) for additional information.

INSTRUCTIONS FOR USE:

1. Apply 2-3 drops (or more if necessary) of Key Surgical Adhesive Remover directly on sticky residue.
2. Let stand until residue softens.
3. Wipe clean with a damp cloth until no residue remains.
4. Wipe with rubbing alcohol and allow surface to dry before reapplying labels or instrument identification tape.

Consult the instrument manufacturer's IFU for instrument reprocessing information.

STORAGE:

Store in a cool, dry place. Keep container tightly closed when not in use.

DISPOSAL INSTRUCTIONS:

Dispose of waste in an appropriate authorized treatment facility in accordance with regulations in force and product characteristics at time of disposal.



MANUFACTURER: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM



CATALOG
NUMBER



LOT
NUMBER



MDSS GMBH
SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY

