

Frequency of Use

Use daily to evaluate the efficacy of air removal and steam penetration in prevacuum steam sterilizers.

Instructions for Use

1. Place a test pack in the front, bottom section of the sterilizer rack, near the door and over the drain, in an otherwise empty chamber.
2. Process at 134°C (273°F) for 3.5–4 minutes. Do not exceed 4 minutes.
3. After processing, carefully remove the AirView™ Test Pack and allow to cool.
4. Confirm that the indicator on the box has changed color and then remove the indicator test sheet from the center of the test pack.
5. The indicator test sheet should show a uniform color change. Any unexpected color change may indicate sterilizer malfunction and should be immediately reported to the supervisor on duty for review.
6. Complete the information on the test sheet and retain as permanent record.

Storage and Shelf Life

Store at controlled room temperature. The AirView™ Test Pack has a three (3) year shelf life from date of manufacture.

AAMI (Association for the Advancement of Medical Instrumentation)
AirView™ is a trademark of SPSmedical Supply Corp.



DISTRIBUTED BY KEY SURGICAL, LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM