

Bite Blocks

INTENDED USE:

The Key Surgical® Bite Blocks are intended to ensure patient's mouth remains open, with teeth protected, during endoscopic procedures and to protect the endoscope.

Product Number	Description	Port Size
EN-320	Adult bite block w/ VELCRO® brand strap	21 mm / 60 FR
EN-310	Adult bite block w/ pliable strap	21 mm / 60 FR
EN-210	Pediatric bite block w/ pliable strap	16 mm / 48 FR
EN-315	Adult O ₂ -CO ₂ bite block w/ pliable strap	21 mm / 60 FR
EN-300	Comfort block (no strap)	16 mm / 48 FR

CONTRAINDICATIONS:

There are no known contraindications and/or adverse effects.

INSPECTION:

Carefully inspect all inner and outer surfaces of the product for cracked surfaces of sharp edges.

Do not use if product is damaged or is not in original individual packaging.

INSTRUCTIONS FOR USE:

- Place bite block in the patient's mouth.
- If retaining strap is used, secure the strap on the bite block.
 - Pliable strap (EN-210, EN-310, EN-315): To secure the retaining strap, thread over cleat on side of bite block.
 - VELCRO® brand strap: (EN-320): Press strips together to fasten and pull apart to unfasten.
 - If no strap is used (EN-300), hold the bite block in place with pressure from hands or fingers during the entire procedure.
- When procedure is complete, and patient is awake, remove the bite block from the patient's mouth.

DISPOSAL:

Bite Blocks should be disposed of after use; in accordance with facility disposal procedure.



MANUFACTURER: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM



CATALOG
NUMBER



MDSS GMBH
SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY



DO NOT RE-USE