

Burr Cleaning Brushes

INTENDED USE:

Key Surgical® Burr Cleaning Brushes are single use brushes specially designed for cleaning surgical instruments during a procedure to keep burrs, rasps, files and saws free of debris. These brushes are designed for use in the operating room.

CONTRAINDICATIONS:

There are no known contraindications and/or adverse effects.

PREPARATION:

Brushes do not need to be cleaned or disinfected prior to use. The brushes can however, be put through a washer if desired.

CLEANING & DISINFECTION:

Brushes can be cleaned manually or using an automatic washer/disinfector.

INSPECTION:

Brushes should be inspected and replaced if worn, frayed, bent or otherwise damaged. Worn or damaged bristles and handles are ineffective in cleaning.

STERILIZATION:

Brushes are provided non-sterile and have been validated for sterilization efficacy using gravity and pre-vacuum steam sterilization methods according to the following parameters:

<i>Cycle Type</i>	<i>Temperature</i>	<i>Minimum Time – Full Cycle</i>
Gravity	121° C (250° F)	30 minutes
Pre-Vacuum	132° C (270° F)	10 minutes
Pre-Vacuum	134° C (273° F)	18 minutes
Pre-Vacuum	134° C (273° F)	3 minutes

STORAGE:

1. After sterilization, move wrapped set to sterile storage area for storage or transport to point of use location.
2. Do not handle roughly or drop.
3. Dispose of brush after use.

DISPOSAL:

There are no special disposal instructions.

 MANUFACTURER: KEY SURGICAL, LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM

REF CATALOG NUMBER

EC REP MDSS GMBH
SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY

LOT LOT NUMBER

 CONSULT INSTRUCTIONS FOR USE

 DO NOT RE-USE

CE