

# ISI DuraHand



LOT # 4560L

## **Sterilization Instructions:**

Device must be cleaned prior to sterilization. The preferred cleaning solution is ISI's Hemaway Tri-Max but if unavailable, any non-acidic, general purpose cleaner suited for standard surgical instruments is acceptable. The DuraHand should then be autoclaved.

Cycle Type: Pre-Vac Steam Sterilization  
Full Cycle Instructions

Double wrapped in standard central supply wrap  
Sterilization Temperature: 132°C  
Sterilization Exposure Time: 4.0 Minutes

Cycle Type: Gravity Steam Sterilization  
Full Cycle Instructions

Double wrapped in standard central supply wrap  
Sterilization Temperature: 121°C

## **DuraHand Instructions for Use:**

Place patient's hand on top the surgical DuraHand

Bend wrist tabs over patient's wrist and finger tips over patient's fingers as needed. The patient's hand should then be flat and immobilized. The tab opposite of the thumb can be used as a "kickstand" to prevent the wrist from rolling.

After procedure, straighten DuraHand fingers and wrist tabs.

DuraHand must be cleaned in accordance with standard institutional protocol prior to sterilization.

Sterilize according to "Sterilization Instructions" provided.

Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

***Sterilize DuraHand prior to use!***

***If the DuraHand coating is cracked,  
discard the device.***

***DO NOT USE!***



DISTRIBUTED BY KEY SURGICAL, LLC  
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA  
PH: 800.541.7995 KEYSURGICAL.COM



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands