

### Endo-Scrub Dry Sponge

**INTENDED USE:**

Key Surgical® Endo-Scrub Dry Sponges are intended to be used to help remove gross soil during pre-cleaning and decontamination.

**CONTRAINDICATIONS:**

There are no known contraindications and/or adverse effects.

**INSTRUCTIONS FOR USE:**

Endo-Scrub Dry Sponge can be used to remove gross soil with or without detergent solution.

1. Place sponge around insertion tube of endoscope.
2. Wipe down the length of the endoscope to remove gross soil.
3. Rinse sponge and repeat until the exterior surface is free from gross soil.

**DISPOSAL:**

Endo-Scrub Dry Sponge should be disposed of after use; in accordance with facility standard practices.



MANUFACTURER: KEY SURGICAL LLC  
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA  
PH: 800.541.7995 KEYSURGICAL.COM



MDSS GMBH  
SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY



DO NOT RE-USE