



## Endoscope Tip Protector

### INTENDED USE:

Key Surgical® Endoscope Tip Protector is intended to protect the distal tip of semi-critical endoscope during transportation and storage.

### CONTRAINDICATIONS:

There are no known contraindications and/or adverse effects.

### INSTRUCTIONS FOR USE:

When used with a disinfected endoscope:

1. Slip the Tip Protector onto the distal end of the endoscope insertion tube after the endoscope has been properly cleaned, disinfected and completely dried.
2. Remove the Tip Protector prior to patient use.

When used with a dirty endoscope:

1. Slip the Tip Protector onto the distal end of the endoscope insertion tube for transportation to the cleaning area.
2. Remove the Tip Protector prior to cleaning, soaking, disinfection/sterilization of the endoscope.

### DISPOSAL:

Endoscope Tip Protector should be disposed of after use; in accordance with facility standard practices.



MANUFACTURER: KEY SURGICAL LLC  
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA  
PH: 800.541.7995 KEYSURGICAL.COM



MDSS GMBH  
SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY

