

Dry Cleaning Wipe (WIPE8X8)

EN - **INTENDED USE:** Key Surgical® Dry Cleaning Wipes are intended to remove soils and clean surfaces.

CONTRAINDICATIONS: No known contraindications and/or adverse effects.

PRECAUTIONS: Store away from a source of ignition. Single use only; reuse may lead to patient cross-contamination.

PREPARATIONS FOR USE: Follow the item manufacturer's instructions for use. If utilizing a cleaning agent, be sure to follow the cleaning agent manufacturer's IFU for proper preparation of the cleaning solution.

INSTRUCTIONS FOR USE:

1. Open the packaging
2. Take out the product
3. Wipe down item or surface to remove residual soils or moisture
4. Dispose after single use

DISPOSAL: Dispose after use according to facility protocol. Do not reuse.



KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 USA
(US) TEL: 800.541.7995 (UK) TEL: +44(0)1628 810626
(EU) TEL: +49 4363 905900

REF

CATALOG
NUMBER

LOT

LOT
NUMBER



MANUFACTURER
FOR US: MANUFACTURED FOR



UNIQUE DEVICE IDENTIFIER



DO NOT RE-USE



DATE OF MANUFACTURE



STERIS®