

### Soaker Sheet Shelf Liners

**EN** - **INTENDED USE:** Key Surgical® Soaker Sheet Shelf Liners (SOK008A, SOK009A, SOK010A) are intended for placement on autoclave shelves to absorb wetness and humidity during processing.

**CONTRAINDICATIONS:** No known contraindications and/or adverse effects.

**PRECAUTIONS:** Store away from a source of ignition. Single use only; reuse may lead to patient cross contamination.

#### PREPARATIONS FOR USE:

1. Select appropriate size product for autoclave shelf
2. Place product on autoclave shelf under the wrapped tray or rigid container system

#### INSTRUCTIONS FOR USE:

1. Sterilize according to the information below
2. Place product on autoclave shelf

#### STERILIZATION:

Evaluated for sterilization compatibility and found to be compatible with steam sterilization. Provided non-sterile and are single use. Not suitable for vaporized hydrogen peroxide/plasma sterilization.

It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

**DISPOSAL:** Dispose after use according to facility protocol. Do not reuse.



KEY SURGICAL LLC  
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 USA  
(US) TEL: 800.541.7995 (UK) TEL: +44(0)1628 810626  
(EU) TEL: +49 4363 905900



MANUFACTURER  
FOR US: MANUFACTURED FOR



CATALOG  
NUMBER



DO NOT RE-USE



LOT  
NUMBER



CONSULT  
INSTRUCTIONS  
FOR USE



CAUTION



DATE OF MANUFACTURE



UNIQUE DEVICE IDENTIFIER



STERIS®