

Suture Booties

EN - **INTENDED USE:** Key Surgical® Suture Booties are disposable, single-use accessories to medical devices intended for use by fully qualified medical personnel to facilitate the grasping and tagging of delicate sutures. They are surgically invasive devices intended for transient-term use (less than 60 minutes); to aid the surgeon's handling of delicate sutures; they are not intended for implantation.

PRECAUTIONS: No precautions for disposal apply. This device is single use only; reuse may lead to patient cross contamination and/or device failure. Do not re-sterilize. Re-sterilization of the device may result in loss of product functionality.

CAUTION: Prior to use visually inspect packaging. Do not use if damaged, opened or breached as sterility cannot be guaranteed.

CONTRAINDICATIONS: Not intended for use in direct contact with the central nervous system.




INSTRUCTIONS FOR USE:

1. After removing pad from pouch, remove backing to expose adhesive and apply to appropriate surface.
2. Insert jaws of instrument into suture booties and remove from pad.
3. Upon completion of procedure, dispose of both used and unused booties according to hospital protocol.

 **KEY SURGICAL LLC**
 8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 USA
 (US) TEL: 800.541.7995 (UK) TEL: +44(0)1628 810626
 (EU) TEL: +49 4363 905900

 **MANUFACTURER FOR US: MANUFACTURED FOR** REF **CATALOG NUMBER** LOT **LOT NUMBER**

 **CONSULT INSTRUCTIONS FOR USE**  **USE-BY DATE**  **DO NOT RE-USE**  **CAUTION**  **DO NOT USE IF PACKAGE IS DAMAGED** **UDI** **UNIQUE DEVICE IDENTIFIER**

STERILE **R**  **DO NOT RESTERILIZE** **MD** **MEDICAL DEVICE**  **SINGLE STERILE BARRIER SYSTEM**  **SINGLE STERILE BARRIER SYSTEM WITH PROTECTIVE PACKAGING OUTSIDE**

